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**GB CARE AND CLEANING:** For cleaning soiled faux leather, use warm soapy water. Then rinse with clear water and wipe the surfaces with a dry cloth. Do not use solvent-based or oil-based cleaning agents. Ether, heating oil, diesel, gasoline, and motor oil can damage the fabric and coating! Since we have no control over possible changes in the ingredients of disinfectants, fluctuations in raw materials, or the method of application, ultraMEDIC GmbH cannot provide any guarantees. It is recommended to test the cleaning agent on an inconspicuous area of the material. **MAINTENANCE:** Please check all screw and plug-in connections at regular intervals and retighten. Loose screw connections will lead to impaired safety! Do not use the chair if this should be the case. **REPLACING/WORKING ON THE PNEUMATIC SPRINGS:** Only qualified personnel with appropriate instruction are permitted to work on or replace the pneumatic springs. **CASTORS AND FLOORS:** Your swivel chair is equipped with braked casters for hard floors as standard. If it is used on insensitive hard floors (e.g. tiles, stone), this measure will suffice. However, if used on sensitive hard floors (e.g. parquet flooring, cork, laminate), you will also have to use a standard commercially available floor mat for protection in addition to the hard floor castors to avoid causing any damage to the floor. We shall accept no liability for any damages in the event of failure to comply with this instruction. **DISPOSAL:** Keep the packaging materials out of reach of children and, in the interests of environmental protection, send them for recycling. When the chair has served its useful time, check with your local authority about how to dispose of it in an environmentally friendly manner. **CAUTION:** Keep all small parts, packaging bags and film away from babies and toddlers because of the risk of suffocation and choking.

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## GB ASSEMBLY INSTRUCTIONS AND INSTRUCTIONS FOR USE

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