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Contents

	Page
1. Product description	8
2. Important information	9
2.1 Warranty terms	9
2.2 User group	9
3. Information on the safety and protection of the product	9
4. Intended use	10
4.1 Fitting ultraGEAR MOBILE to the basket stretcher	10
4.2 Fitting ultraGEAR II to the basket stretcher	11
5. Cleaning and care	11
6. Maintenance	11
7. Storage	12
8. Service life	12

1. Product description

		
<p>Art.-No. SAN-0085-G</p> <p>Article Designation ultraGEAR</p>	<p>Art.-No. SAN-0085-01</p> <p>Article Designation ultraGEAR MOBILE</p>	<p>Art.-No. SAN-0085-02</p> <p>Article Designation ultraGEAR II</p>
<p>Product description</p> <p>A chassis, the wheel axle is firmly welded to the base plate, must be fitted to the basket stretcher ex works.</p>	<p>Product description</p> <p>Fully removable and tool-free wheel system with a thru axle.</p>	<p>Product description</p> <p>Factory pre-assembled base plate for attaching 2 side wheels via a quick-release axle.</p>
<p>Compatible</p> <p>only with ultraMEDIC basket stretchers ultraBASKET STRETCHER: SAN-0087-1 (1-piece) or SAN-0087-2 (2-piece)</p>	<p>Compatible</p> <p>only with ultraMEDIC basket stretchers ultraBASKET STRETCHER: SAN-0087-1 (1-piece) or SAN-0087-2 (2-piece)</p>	<p>Compatible</p> <p>only with ultraMEDIC basket stretchers ultraBASKET STRETCHER: SAN-0087-1 (1-piece) or SAN-0087-2 (2-piece)</p>
<p>Load capacity</p> <p>200 kg</p>	<p>Load capacity</p> <p>150 kg</p>	<p>Load capacity</p> <p>200 kg</p>
<p>Weight</p> <p>3,9 kg</p>	<p>Weight</p> <p>6,36 kg</p>	<p>Weight</p> <p>8,1 kg</p>
<p>Dimensions LxW</p> <p>660 x 270 mm</p>	<p>Dimensions LxW</p> <p>660 x 270 mm</p>	<p>Dimensions LxW</p> <p>660 x 270 mm</p>
<p>Scope of delivery</p> <p>1 unit, pre-assembled at the factory</p>	<p>Scope of delivery</p> <p>1 base plate 2 wheels with quick-release axle 2 straps with push-button lock 2 folding pins</p>	<p>Scope of delivery</p> <p>1 base plate pre-assembled at the factory 2 wheels with quick-release axle 2 folding pins</p>
<p>Spare parts</p>	<p>Spare parts</p> <p>MAT-0085-KS Folding pin for ultraGEAR</p> <p>SAN-0084-MO Fastening strap for ultraGEAR MOBILE 2-piece with push-button lock and hook</p>	<p>Spare parts</p> <p>MAT-0085-KS Folding pin for ultraGEAR</p>

2. Important information

Before using the product, please read the instructions for use and safety instructions carefully. Keep the instructions for use in a safe place to avoid operating errors.

Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealership or ultraMEDIC GmbH directly.

2.1 Warranty terms

The manufacturer grants a warranty period of 24 months on this product. The warranty only encompasses the costs of the spare parts.

The following points are not covered by the warranty:

- possibly omitted shipping or labour costs incurred on the purchaser's side;
- defects in the product due to normal wear and tear;
- damages caused by the user;
- effects or damages caused by non-compliance with the instructions in this document;
- any modifications or repairs to the product without prior approval by ultraMEDIC GmbH.

2.2 User Groups

This product may only be used by qualified, expert personnel. A prerequisite is theoretical and practical experience in handling the product.

3. Information on the safety and protection of the product

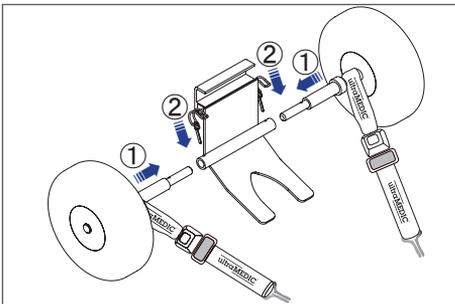
1. Before and after every use, check the components of the product for damage and completeness.
2. Ensure that the product is correctly installed.
3. Before initial use, conduct a test run with an empty stretcher to become aware of the handling of the product.
4. In the event of damage or suspected damage to the product, remove the product from operation immediately until any occurring irregularities have been remedied.


4. Intended use

All 3 ultraGEAR models have been developed to relieve rescuers during their missions. The innovative design significantly reduces the weight to be carried, which reduces the physical strain on rescuers and increases efficiency. Especially on flat surfaces, our product enables effortless maneuvering over longer distances. The user-friendly design ensures that even longer transport distances can be covered with ease.

4.1. Fitting ultraGEAR MOBILE to the basket stretcher

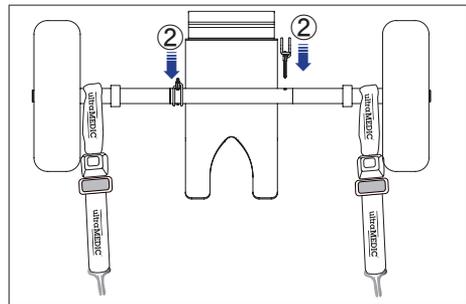① Connect the base plate to the quick-release axles of the tires. Make sure that the holes in the thru axle and the base plate are aligned.

② Fasten the quick-release axles to the base plate on both sides using folding pins.

③ Attach the fully assembled base plate to the edge of the basket stretcher by flange-mounting the base plate bracket to the edge of the basket stretcher. Please always use the foot side of the basket stretcher.

④ Hook the straps into the recessed grips and tighten.

4.2. Fitting ultraGEAR II to the basket stretcher

To mount ultraGEAR II on the basket stretcher, follow steps 1 and 2 from point 4.1.

5. Cleaning and Care

You can use commercially available alcohol-based agents for disinfection. Other solutions or cleaning chemicals can damage the material.

Straps: Straps that have become wet should only be allowed to dry naturally and should be kept away from direct heat.

Metal parts: Clean with water, neutral soap or a suitable disinfectant, rinse and dry.

6. Maintenance

In order to ensure the safety of the user and the injured party during a rescue mission, we recommend that you have the product visually inspected by an expert on an **annual** basis.

For users outside of Germany, a regular check according to country-specific requirements is required.

The following parts must be checked and the following defects must be excluded:

Base plate	Deformation, tears, breaks, loosened screws, broken parts
Straps	Cutting and abrasion points, damage to the seams, tissue-damaging stains
Metal parts	Rust, oxidation, deformation
Plastic parts	Deformation, tears, breaks
other damages	

In the event of damage or suspected damages, remove the product from operation immediately until any occurring irregularities have been remedied.

7. Storage

Store the product in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors can damage the material.

Protect the moving parts from dust and dirt deposits to maintain their maximum effectiveness during use.

8. Service life

The service life of the product depends on the individual conditions of use. Depending on how frequently and intensely the product is used, wear and tear on the product is normal. In this regard, correct use is assumed. The material is subject to an ageing process even if handled carefully. The product should be discarded depending on its condition.

The service life of the **straps** (model SAN-0085-01) is set at a maximum of **10 years** after commissioning.

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