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## ultraSAVER EVAC

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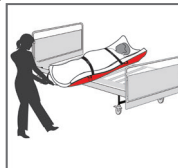
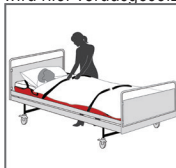
# INSTRUCTIONS for USE

## ultraSAVER EVAC

Evacuation sheet with straps and corner elastics

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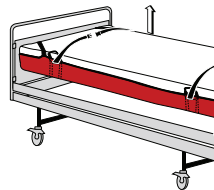
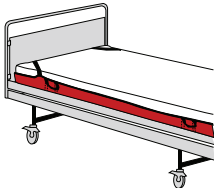
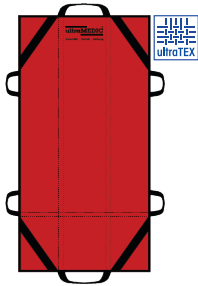
Always up to date:  
Instructions for use online

## Intended use of the product

The ultraSAVER EVAC is a medical device. **Indication:** Suitable for the rescue and evacuation of bedridden or injured persons. **Contraindication:** Not suitable for the transport of patients with neck and spinal injuries.

## User groups

According to § 2, paragraph 2 MPBetriebV, this product may only be used by competent, trained specialists. The prerequisite is theoretical and practical knowledge of how to use the evacuation sheet.

## Important information

Before using the evacuation sheet, please read the instruction for use and safety instructions carefully. Keep the instruction for use in a safe place to avoid operating errors.

## Declaration of conformity

This product was manufactured in accordance with the current EU regulation 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

## General safety instructions

Check the integrity of the evacuation sheet **before and after** each use. If damage or only suspicion of damage occurs, take the product out of operation immediately. All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

## Use

Place the ultraSAVER EVAC under the mattress and fasten it to the corners of the mattress with the corner elastics. A bed sheet can then be pulled onto the mattress as usual. However, care must be taken to ensure that the belts are easy to reach for the nursing staff. The straps for fixing the bedridden person can be stowed in the pockets.

Secure the bedridden person with the two restraint straps stowed in the pockets. Make sure that the fixation straps are tight but do not cut into you. Carefully lift the evacuation sheet and mattress off the bed, head first.

## Storage

Store the evacuation sheet in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors can damage the material.

## Maintenance and Care

The ultraSAVER EVAC can be used several times. The evacuation sheet is suitable for spray and wipe disinfection.

To ensure the safety of the patient and the user, we recommend that the rescue sheet be subjected to **an annual** visual inspection by an expert.

For users in Germany, we recommend observing the following regulations and guidelines: DGVU Principle 305-002 / DGVU Rule 112-198/199 / MPBetriebV.

## Service life

The service life of the ultraSAVER EVAC depends on the individual conditions of use. Depending on the frequency and intensity of use certain wear and tear is normal. Appropriate use is assumed here.

The material is subject to aging even when handled carefully. This is accelerated by UV rays, heat, chemicals and dirt. The product should be phased out depending on the condition or no later than 10 years after commissioning. Proper usage is assumed.