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Content

	<i>Page</i>
1. Product description	2
1.1 Technical data	2
1.2 Intended use of the product	8
1.3 User groups	8
1.4 Customer service	8
1.5 Declaration of conformity	8
1.6 Functions	8
2. Liability and warranty guarantee	8
3. Important Notes	9
4. Application	10
4.1 Unfolding the carrying and rescue chair	10
4.2 Folding the carrying and rescue chair	11
4.3 Attaching the patient belts	11
5. Cleaning and Care	11
6. Maintenance	12
7. Storage	12
8. Service life	12
9. Identification and Warranty Certificate	13

Before using the ultraRESCUE-CHAIR, please read the instructions for use and safety instructions carefully. Keep the instructions for use in a safe place to avoid operating errors.

1.2 Intended use of the product

The ultraRESCUE-CHAIR (carrying and rescue chair) is a medical device.

Indication: Suitable for the safe transport of patients, for example, in hospitals, emergency centers, stairwells, and fire rescue operations.

Contraindication: The rescue chair is not suitable for prolonged sitting.

1.3 User groups

This product may only be used by qualified, expert personnel. A prerequisite is theoretical and practical experience in handling the carrying and rescue chair.

1.4 Customer service

Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealership or ultraMEDIC GmbH directly.

Always include the item and LOT number to identify the product. You can find the article number on the product label. The LOT number sticker is located next to the product identification label. The label and the sticker must not be removed, this leads to the loss of the guarantee, among other things.

1.5 Declaration of conformity

The ultraRESCUE-CHAIR was manufactured in accordance with the current EU regulation 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

1.5 Functions

The carrying chair is foldable and can be stowed away to save space. The four wheels that the chair is equipped with make it easy to transport patients. The chair is made of very strong aluminum alloy materials, which makes it light, small and convenient to carry. The stretcher chair is equipped with safety belts to ensure patient safety during transportation. Removable covers make it easier to clean and sterilize the surfaces.

2. Liability and warranty guarantee

The manufacturer grants a warranty period of 24 months on this product. The warranty

only encompasses the costs of the spare parts.

Any shipping or labor costs incurred are not guaranteed. The following points are not covered by the warranty:

- transportation,
- operator error,
- incorrect, improper or no maintenance,
- damage caused by incorrect operation of the product,
- repairs carried out by unauthorized persons,
- damage caused by use in unsuitable conditions,
- normal wear and tear.

3. Important Notes

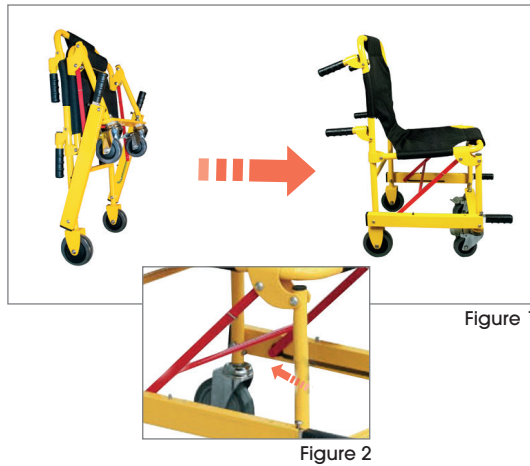
1. Before initial use, conduct a test run with the ultraRESCUE-CHAIR to become aware of the handling of the product.
2. The carrying chair may only be used when it is unfolded!
3. Before the patient is transported, an initial specialist medical assessment must be carried out.
4. Make sure that the ultraRESCUE-CHAIR is correctly assembled.
5. Ensure that the patient restraining belts are correctly installed.
6. Before transport, the patient must be secured using patient belts. Insufficient fixation or no fixation can lead to serious injuries of the patient and the user.
7. Before and after every use, check the components of ultraRESCUE-CHAIR and patient restraining belts for damage and completeness.
8. In the event of damage or suspected damage to the ultraRESCUE-CHAIR or belts, remove the product from operation immediately until any occurring irregularities have been remedied.
9. All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

2. Applikation

4.1 Unfolding the carrying and rescue chair

To transport the patient, first unfold the carrying and rescue chair (Figure 1), then pull the locking bar in the direction shown (Figure 2) until it locks into place.

Place the patient on the carrying and rescue chair and secure with safety belts.

Pull out the telescoping handles as needed and open the folding handles before use (Figure 3).

Figure 3

4.2. Folding the carrying and rescue chair

To fold the carrying and rescue chair for storage, pull the locking bar as shown in Figure 4. To do this, release the lock. Then the chair can be folded together.

Figure 4

4.3. Attaching the patient belts

The scope of delivery includes 3 patient belts to secure the patient on the carrying and rescue chair. Attach the patient straps to the frame of the stretcher (Figure 5). The patient's torso is secured to the seat with 2 crossed straps, 1 strap is used to fix the legs to the carrying chair.

Figure 5

5. Cleaning and Care

The product must not be cleaned with a high-pressure/steam jet! This could lead to de-lubrication and corrosion.

Use mild soap and warm water to clean the carrying and rescue chair and belts. You can use commercially available alcohol-based agents for disinfection. Other solutions or cleaning chemicals may damage the material of the product and the belts.

6. Maintenance

In order to ensure the safety of the user and the injured party during a rescue mission, we recommend that you have the ultraRESCUE-CHAIR and belts visually inspected by an expert on an **annual** basis.

For German users, we recommend consideration to the following regulations and guidelines:

- MPBetreibV
- DGUV Grundsatz 305-002 DGUV Regel 112-198/199
- Keep routinely clean (incl. disinfection)

For users outside of Germany, a regular check according to country-specific requirements is required.

In the event of damage or suspected damages, remove the product from operation immediately until any occurring irregularities have been remedied.

The **screw connections** on the carrying and rescue chair must be checked regularly. If the pin nut and steel screws loosen, they must be tightened.

The **moving parts** of the carrying and rescue chair (connecting and rotating parts) should be oiled in the event of a malfunction or blockage.

The **patient belts** must be checked for integrity after each use.

7. Storage

Store the ultraRESCUE-CHAIR in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors can damage the material.

8. Service life

The service life depends on the condition of the product, its frequency of use, and the external operating conditions. The lifespan of the ultraRESCUE-CHAIR is dependent on individual usage conditions. Depending on the frequency and intensity of use, some wear and tear is normal. Even with careful handling, the material is subject to an aging process. The product should be decommissioned after an assessment of its condition. Proper use is assumed here.

The lifespan of the straps is set at a maximum of 10 years from commissioning. The lifespan of optional accessories can be found in the respective user manuals.

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