SKYLOTEC

Click Me

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CONREST PRO+

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
Instruction for use	<u>GB</u>
Gebrauchsanleitung	<u>DE</u>
Istruzioni d'uso	<u>IT</u>
Instructions d'utilisation	<u>FR</u>
Instrucciones de uso	<u>ES</u>
Instruções de serviço	<u>PT</u>
Gebruiksaanwijzing	<u>NL</u>
Brugsanvisning	<u>DK</u>
Bruksanvisning	<u>NO</u>
Käyttöohjeet	<u>FI</u>
Bruksanvisning	<u>SE</u>
Οδηγίες χρήσης	<u>GR</u>
Talimatlar	<u>TR</u>
Instrukcje	<u>PL</u>
Navodila	<u>SL</u>

SKYLOTEC GmbH
Im Mühlengrund 6-8
56566 Neuwied · Germany
Fon +49 (0)2631/9680-0
Mail info@skylotec.com
Web www.skylotec.com

(EU) 2016/425

© SKYLOTEC
MAT-BA-0190-02
Stand 10.10.2024


Informationen/Information

Content	
Icons	5
GB	Instruction for use 9
DE	Gebrauchsanleitung 14
IT	Istruzioni d'uso 19
FR	Instructions d'utilisation 24
ES	Instrucciones de uso 29
PT	Instruções de serviço 34
NL	Gebruiksaanwijzing 39
DK	Brugervejledning 44
NO	Bruksanvisning 49
FI	Käyttöohjeet 53
SE	Bruksanvisning 58
GR	Οδηγίες χειρισμού 63
TR	Talimatlar 68
PL	Instrukcja użytkowania 73
SL	Navodila 78


1.) Rescue system/ Rettungssystem

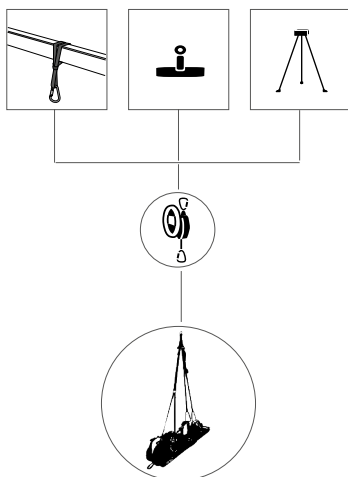Icons

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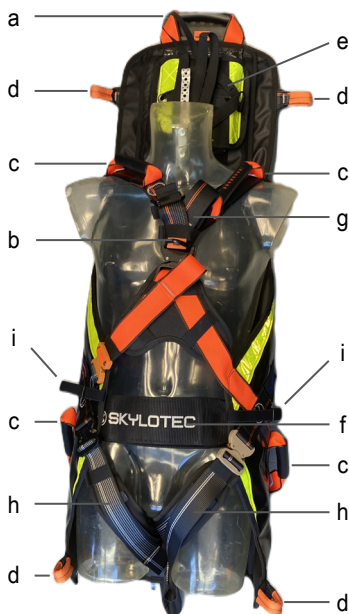
2.1

2.3

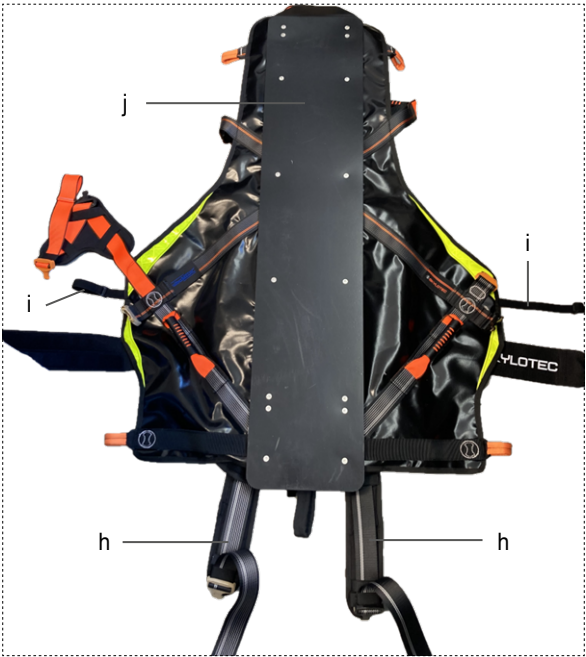CONREST PRO+ front view /
CONREST PRO+ Vorderansicht

CONREST PRO+ rear view / CONREST PRO+ Rückansicht

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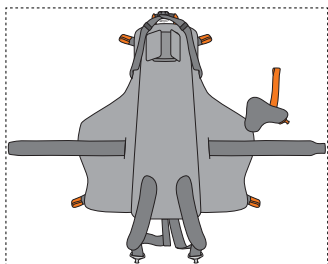
8.


1.

4.) How to put on the CONREST PRO+ / Anlegen der CONREST PRO+

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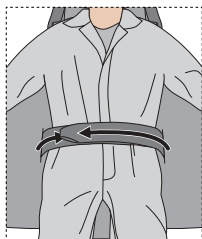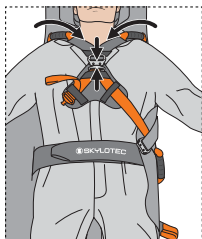
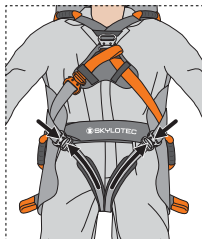
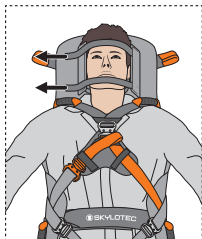
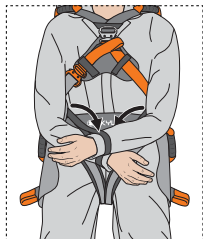
5.4

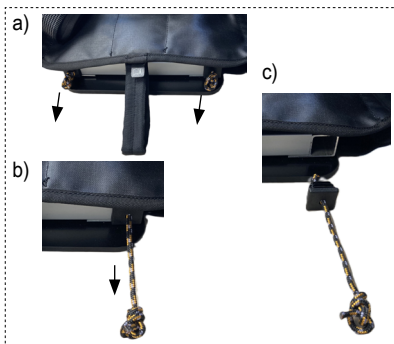
5.5

6.) Attachment of the leg support / Befestigung des Beinsupports

6.1

6.2

General information - read carefully

Read and understand this manual before using the product.

The instructions must always be available in the national language. If these are not available, the seller must clarify this with SKYLOTEC before resale. The instructions must be made available to the user. The equipment may only be used by persons in good physical and mental health. They must be trained in safe use and have the necessary knowledge, or be under the supervision of such a person. Emergency and evacuation plans must be in place for all eventualities. Rescue operations must be able to be carried out as quickly as possible.

1.) Rescue system

A rescue system consists at least of a suitable body-holding device, e.g. a rescue harness according to EN 1497, a rescue lifting device EN 1496 or abseiling device EN 341 and an anchor point, e.g. according to EN 795. Only tested and approved components may be used within the described conditions of use and for the intended purpose. When combining individual components, it must be ensured that the safe function of each component and of the assembled rescue system is always guaranteed, as there is a risk to life and limb in the event of non-compliance.

The anchor point to which the equipment is attached must have a minimum strength of 12 kN (approx. 1.2 t). Select/position anchor points as vertically as possible, above the person to be rescued. When connecting directly to structural equipment, make sure that no sharp edges endanger the equipment and that the connection cannot come loose by itself. In particular, make sure that all EN 362 connectors (carabiners) are securely fastened.

2.) Designation of the parts

2.1 / 2.2 Rescue harness

- a. Carrying handle / attachment point EN 1497
- b. Attachment point EN 1497
- c. Carrying handle / all 4 carrying handles - attachment point EN 1497
- d. Orange loops / all 4 loops - attachment point EN 1497
- e. Movable head fixation / headbands
- f. Hip belt
- g. Shoulder straps
- h. Leg straps
- i. Elastic hand straps
- j. Sliding plate

2.3 Leg support

- k. Velcro Leg strap
- l. Carrying handle
- m. lanyard

3.) Product labelling

- 1. manufacturer incl. address
- 2. size
- 3. observe instructions

-
4. article description
 5. relevant standards + year of issue
 6. serial number
 7. CE marking of the supervising body
 8. month and year of manufacture
 9. QR code
 10. internal barcode
 11. item number
 12. max. Number of persons
 13. max. Nominal load incl. equipment

The CONREST PRO+ rescue harness/stretcher has been developed for emergency evacuation from confined spaces and must not be used as a body-holding device in a fall arrest system. The max. rated load must not exceed 150 kg. Take care to avoid suspension trauma due to hanging for too long.

4.) How to put on the CONREST PRO+

- Open all straps and click locks and put them aside (4.1).
- Place the injured person on the CONREST PRO+ and close the Velcro hip belt (4.2).
- Pull the shoulder straps over the head and fasten the black buckle with attachment point (4.3).
- Close the orange click lock (4.4).
- Close the click locks of the leg loops, observing the colours (black, beige). Adjust the leg loops so that the legs are not cut off (4.5).
- Securely fasten the head using the Velcro straps (4.6).
- Place the hands in the elastic loops to secure the arms (4.7).

The webbings should be adjusted so that two fingers fit between the webbing and the person.

5.) Attachment of the rescue system

The attachment points in the head area 2.1 a and in the chest area 2.1 b can be used as independent attachment points during use. (See 5.1 and 5.2 for application).

Note: Ensure that the rescue system is centred at the attachment point in the head area (5.1)!

The CONREST PRO+ has 4 handholds (2.1 c), which are marked with „bridles“. These four handholds can be used in combination with a carrying system (SKYLOTEC UltraBridles) as attachment points. (For application see figure 5.3).

The CONREST PRO+ additionally has 4 orange loops (2.1 d), which can also be used as attachment point with a carrying system (SKYLOTEC UltraBridles). (See Figure 5.4 for application).

For horizontal and/or vertical positioning of the injured person, the lower 2 orange loops can be connected with the attachment point in the chest area using the Triangle Sling (Article No. SAN-9106). (See Figure 5.5 for application).

This ensures safe transport of the injured person and there is no risk of injury.

6.) Attachment of the Leg Support (optional accessory, part number SAN-9105)

- Remove plugs from the bottom of the aluminum profiles (6.1)
- Insert the leg support from below into the aluminum profiles until it audibly clicks into place (6.2)
- Secure the legs of the injured person with the Velcro leg strap (k)
- If necessary, connect the lanyard (l) on the leg support to the carrying system and adjust the length

Caution: Do not adjust the lanyard (l) too tightly to prevent the leg support from bending.

- **Note:** Do not use the carrying handles (k) on the leg support for the carrying systems!

7.) Safety requirements

The instructions must be strictly observed for the protection of the user and the equipment! The product labelling must be completely legible! To avoid suspension trauma, an emergency plan must be in place to keep the suspension time in the harness or loop as short as possible, thus preventing a circulatory collapse. There is an essential need for the user to check all attachment and/or adjustment parts regularly during use.

- Application temperature -35 to +45°C
- Store and transport dry and protected from light
- Avoid slack rope
- Avoid contact with aggressive substances
- Observe danger from suspension trauma
- Observe danger from edges and rough surfaces
- Observe danger from electrical equipment
- Observe danger from moving machine parts
- Do not wear PPE in areas where there is a risk of falling.
- Avoid heat and flames.

8.) Changes, repairs

Any changes or additions must not be made by the user, but exclusively by SKYLOTEC.

Damaged and/or defective components (e.g. the sliding plate) may only be replaced by SKYLOTEC or a service company trained by SKYLOTEC!

9.) Lifetime

The max. lifetime (period between production date and discard date) results from the storage period before delivery to the end consumer and the period of use.

With the storage period of max. 2 years before delivery to the end user or before purchase, it should be noted that the products are

- stored without extreme temperature fluctuations,
- protected from UV radiation, moisture, chemicals and harmful/aggressive environmental conditions, and
- stored in the original undamaged packaging.

The service life begins with the delivery to the end user and is max. 10 years. After delivery to the end user (proof by e.g. purchase

receipt/delivery note with serial/batch number), regular inspection by a competent person is required at least once a year. In addition, the respective country-specific regulations must be observed, which may prescribe shorter periods.

Irrespective of the max. lifetime, the shelf life depends on the condition of the product, its frequency of use and the external conditions of use. Every PPE loses durability in the course of its lifetime. The lifetime is determined by use, thermal, chemical, mechanical and harmful/aggressive influences.

10.) Inspection and maintenance

The manufacturer's product liability does not extend to property damage or bodily injury that can occur even with proper function and proper use of equipment. The manufacturer's extended product liability does not apply in the event of modifications to the equipment or failure to observe these instructions or the applicable accident prevention regulations. The safety of the user depends on the effectiveness and durability of the equipment. Check the product for proper functioning before and after each use. Check the complete equipment incl. all individual parts for damage due to deformation, cracks, wear, other changes and for legibility of the labelling. (The regular inspections may only be carried out by the manufacturer or a trained specialist in accordance with the manufacturer's instructions. The inspection intervals depend on the frequency of use and the external conditions of use (dust, heat, etc.) If there is any doubt about safe use, the product must be withdrawn from use immediately and inspected by a specialist. If it is no longer usable, it must be disposed of. Webbing and ropes can be cleaned with warm water (40°C) and mild soapy water. Then rinse well with clear water. Equipment that has become wet must only be allowed to dry naturally and kept away from direct heat.

- Hand wash to max. 40°C.
- Dry and store at room temperature.
- Do not machine clean or dry, do not bleach.
- Avoid contact with aggressive substances/chemicals.
- Marking only permitted on product labels.

Please contact the manufacturer in case of any doubts. If faults are found during the inspection, the product must be replaced.

11.) Storage and transport

Store the product in a dry place protected from direct sunlight. Incorrect storage can have a negative effect on the lifetime of the product! Transport the product/components in suitable containers, protected from direct sunlight and stress, to prevent damage.

12.) Declaration of Conformity

The full Declaration of Conformity can be accessed via the following link: www.skylotec.com/downloads

13.) Documentation

For each product a documentation is required that shall include following information:

Individual information

- Manufacturer + contact details
- Product
- Type/model
- Serial-No.
- Date of production
- Date of purchase
- Date of first use

Regular inspections and maintenance

- Date
- Name and signature of the competent person
- Reason for inspection
- Information on work carried out
- Date of next inspection

14.) List of certifying and production surveillance bodies

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NB 0123: TÜV SÜD Product Service GmbH
NB 0158: DEKRA Testing and Certification GmbH