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Contents

	Page
1. Product description	3
2. Important information	10
2.1 Liability and warranty	10
2.2 User group	10
2.3 Declaration of conformity	10
3. Information on the safety and protection of the product	10
4. Intended use	11
4.1 Cautions	11
4.2 Application as scoop stretcher	11
4.3 Application as spine board	12
5. Cleaning and care	13
6. Maintenance	13
7. Storage	14
8. Service life	14
9. Optional accessories	14
10. Identification and warranty certificate	15

2. Important information

Please read the instructions for use and safety instructions carefully before you start using the COMBI-STRETCHER. Keep the instructions for use in a safe place to avoid operating errors.

Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealer or ultraMEDIC GmbH directly. Always quote the LOT- or serial number to identify the COMBI-STRETCHER.

2.1 Liability and warranty

The manufacturer grants a warranty period of 24 months on this product. The warranty only covers the costs of the spare parts. Any incurred shipping or wage costs are not subject to warranty.

Please check the shipment for damage and completeness immediately upon opening. If damage is found, this must be reported within two working days, packaging damage must be reported immediately. After this period, no guarantee or warranty claims for transport damage can be asserted. No responsibility is taken for improper assembly.

2.2 User group

This product may only be used by competent, trained and qualified personnel, and the principles of use must be observed. A prerequisite for use is theoretical and practical experience in handling the COMBI-STRETCHER.

2.3 Declaration of conformity

The ultraCOMBI-STRETCHER was manufactured in accordance with the current EU regulation 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

3. Information on the safety and protection of the product

1. Before and after every use, check that the components and safety belts are intact and complete.
2. Make sure that the safety belts and components are assembled correctly.
3. Before using the COMBI-STRETCHER for the first time, conduct a test run

to familiarize yourself with the product.

4. In the event of damage or suspected damage to the safety belts or COMBI-STRETCHER remove the product from operation immediately until any occurring irregularities have been remedied

4. Intended use

The ultraCOMBI-STRETCHER is a combined spine board and scoop stretcher used to immobilise injured persons who may have suffered a spinal fracture.

4.1 Cautions

1. A specialist medical assessment must be conducted before the injured party is moved, lifted or transported.
2. To ensure the patient's safety, at least 2 users are required to operate the COMBI-STRETCHER.
3. The injured party must be immobilised on the COMBI-STRETCHER during every use. Insufficient or no restraint may lead to severe patient or rescuer injuries.
4. Serious incidents that have occurred in connection with the product must be reported by the user and / or patient to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

4.2 Application as scoop stretcher

1. To position the ultraCOMBI-STRETCHER underneath the patient without having to move him, open one of the metal fasteners. Unlock the snap buckle by activating the lock mechanism (Figure 1).

Figure 1

There are various application techniques for the scoop stretcher. You can place the halves under the patient from the sides or only open the scoop stretcher at the bottom end, which then forms an "V". Carefully position the top end under the patient's head and close the bottom end (with a so-called

"scooping" motion).

The type of application must be individually determined, depending on the nature of the injury.

2. If the injured person is shorter than 1.20 m, we recommend using the straps SAN-0281 or SAN-0286.
3. Once the parts have been locked (clearly audible clicking sound during the locking process), ensure that both snap locks of the ultraCOMBI-STRETCHER are locked and only then secure the patient with the straps.
4. Secure all 3 patient safety straps. (Figure 2 shows an example of the diagonal fastening.) The latches must always audibly lock into place. For safety reasons, subject the latches to a test by pulling at the appropriate point. Ensure that the injured party's breathing is not encumbered by the traction of the straps.
5. Check all buckles and straps before lifting and transporting the patient.

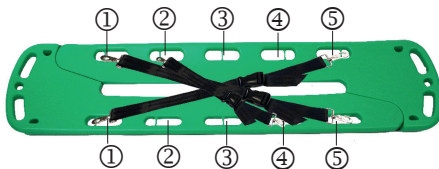
Figure 2

Diagonal fastening:

Strap 1 und 2: Recessed grips ① und ⑤

Strap 3: Recessed grips ② und ④

4.3 Application as spine board

1. Before using the ultraCOMBI-STRETCHER as a spine board, ensure that both snap buckles are locked.
2. Similar to a spine board, the ultraCOMBI-STRETCHER is applied depending on the injury and local conditions.
3. If the injured person is shorter than 1.20 m, we recommend using the straps SAN-0281 or SAN-0286.
4. Secure all 3 patient safety straps. (Figure 2 shows an example of the diagonal fastening.) The latches must always audibly lock into place. For safety reasons, subject the latches to a test by pulling at the appropriate point. Ensure that the injured party's breathing is not encumbered by the traction of the straps.

5. Check all buckles and straps before lifting and transporting the patient.

5. Cleaning and care

Use mild soap and warm water (40°C) to clean the COMBI-STRETCHER.

For disinfection, you can use commercially available, alcohol-based agents. Other solutions or cleaning chemicals may damage the material of the straps, carabiner hooks and clasps.

Allow the wet transportation suspension to dry naturally and keep it away from direct heat exposure.

6. Maintenance

In order to ensure the safety of the injured party and the user, we recommend that you have the COMBI-STRETCHER visually inspected by an expert at least once a year. Appointed auditors must be demonstrably adequately qualified.

For users in Germany, we recommend consideration of the following regulations and guidelines:

DGUV Grundsatz 305-002
DGUV Regel 112-198/199
MPBetreibV

The following parts must be checked and the following defects must be excluded:

Base plate: deformation, tears, breaks, loosened screws, broken parts

Closures: impaired function, deformation, torsion, tears and breaks

Straps: Cutting and abrasion points, damaged seams, tissue-damaging stains

Metal parts: Rust and oxidation

other damages

In the event of damage or suspected damage, remove the product from operation immediately.

7. Storage

Store the COMBI-STRETCHER in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors could damage the material.

8. Service life

The discard criteria depends on the product, its frequency of use and the external conditions of use. The service life of the COMBI-STRETCHER depends on the individual conditions of use. Depending on the frequency and intensity of use of the transportation suspension, certain wear and tear is normal. Appropriate use is assumed here.

Service life of the **patient safety belts (SAN-0296-G)** and of the **safety strap systems (SAN-0286 / SAN-0281)** is set at max. **10 years**.

The service life of the optional accessories can be found in the associated instructions for use.

9. Optional accessories

SAN-0284	ultraHEAD-LOCK I	Head restraining set
SAN-0285	ultraHEAD-LOCK II	Head restraining set
SAN-0286	ultraSTRAPS Xtreme	Safety strap system with clip locks
SAN-0281	ultraSTRAPS	Safety strap system with Velcro fastener
SAN-0280-T	ultraSPINE BAG	Storage bag
SAN-0296-G	ultraBELT	Replacement safety belt
SAN-0260-2	X-BOON TWO	Vacuum mattress
SAN-0278	ultraVACPUMP I	Vacuum pump

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