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Krankentragen ultraSTRETCHER 0350 / 0350 M / 0710 / 0710M / COMPAKT

**ultraMEDIC**

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Stretcher ultraSTRETCHER 0350 / 0350 M / 0710 / 0710M / COMPAKT

**ultraMEDIC**<sup>®</sup>

**Important Information:** Before using the stretcher, please carefully read the operating manual and safety instructions. Keep the operating manual in a safe place to avoid operating errors.

**The product's intended purpose:** The stretcher is used for the transport of the ill and injured. It is widely used within industry, construction sites, first aid services etc.

**Declaration of conformity:** This product was manufactured in accordance with the current EU regulation 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

**Information on the safety:** The stretcher should be used only by instructed persons. The use of this type of products is part of the training program of rescue services.

- The stretcher must not be used for the transport of goods. The approved load is solely for the weight of people.
- The stretcher must not be lifted before the patient restraint straps are correctly closed.
- Serious incidents that have occurred in connection with the product must be reported by the user and / or patient to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

**CAUTION!** Protect the restraint straps from being walked on!

**Handling:** For reasons of space saving can stretchers be folded in length or length and width.

- When folding the stretcher in and out please be careful not to pinch the fabric between the metal parts.
- Adjustable of head- and foot-rest requires checking of the set-pins - they must fit into the setting holes securely.

**Attention!** Unfolding the hinges may squeeze in your fingers!

The stretcher is ready for use when the covering fabric is tensioned and securing pins on the hinges are arrested.

**Cleaning:** After use the stretcher is to be cleaned with water and common cleaning aids and followed sterilized. Only BGA approved aids are allowed. Ether, fuel, petrol and mineral oils may destroy the fabric cover.

**Maintenance:** In order to ensure the safety of the user and the injured party, we recommend that you have the stretcher and the straps visually inspected by an expert on an annual basis.

*In the event of damage or suspected damage, remove the product from operation immediately until any occurring irregularities have been remedied.*

**Repair:** Necessary repair is allowed only by the manufacturer and only OEM parts are to be used.

**Warranty:** The period of warranty starts with reception by the user. Manufacturing and material faults are covered but not damages by inappropriate handling.

<b>1x foldable</b>		
<b>ultraSTRETCHER 0350</b>		
REF	SAN-0101-4G (4 slide feet)	
REF	SAN-0101-4R (4 casters)	
REF	SAN-0101- 2G2R (2 casters / 2 slide feet)	
REF	SAN-R0101-2G2R (red, 2 casters / 2 slide feet)	
REF	SAN-R0101-4G (red, 4 slide feet)	
<b>ultraSTRETCHER 0350 M</b>		
REF	SAN-3101-4G	
REF	SAN-3101-4R	
REF	SAN-3101-2G2R	
<b>2x foldable</b>		
<b>ultraSTRETCHER 0710</b>		
REF	SAN-0104-4G	
REF	SAN-0104-4R	
REF	SAN-0104- 2G2R	
REF	SAN-R0104-4G	
<b>ultraSTRETCHER 0710 M</b>		
REF	SAN-3104-4G	
REF	SAN-3104-4R	
REF	SAN-3104-2G2R	
<b>3x foldable</b>		
<b>ultraCOMPAKT STRETCHER</b>		
REF	SAN-3106-4G	
	<b>150 kg</b>	