**Gebrauchsanweisung DE**  
**Instructions for use EN**  
**Manual de instrucciones ES**

**ultraBASKET STRETCHER**

Korbtrage  
Basket Stretcher  
Camilla cesta

SAN-0087-1  
SAN-0087-2  
SAN-0087-1-M  
SAN-0087-2-XXL

---

# ultraBASKET STRETCHER

Gebrauchsanweisung DE.....	1
Instructions for use EN.....	9
Manual de instrucciones ES.....	17

## Inhalt:

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## Contents:

	Page
<b>1. Product description</b> .....	<b>10</b>
<b>2. Important information</b> .....	<b>11</b>
2.1. The product's intended use .....	11
2.2. Customer service .....	11
2.3. Declaration of conformity .....	11
2.4. Liability and Warranty .....	11
2.5. User group .....	11
<b>3. Information on the safety and protection of the product</b> .....	<b>11</b>
<b>4. Installation</b> .....	<b>12</b>
4.1. Assembling or dismantling the two-piece basket stretcher .....	13
4.2. Optional accessories: ultraBELT X-TREME strap system (SAN-0089) .....	14
4.3. Attaching the ultraBELT X-TREME strap system ultra to the basket stretcher .....	14
<b>5. Application</b> .....	<b>14</b>
5.1. Safety information .....	14
5.2. Positioning the injured party on the basket stretcher .....	14
5.3. Attaching the footrest. ....	14
5.4. Operating the ultraBELT X-TREME strap system ultra on the basket stretcher (optional accessories) ...	15
5.5. Transporting the basket stretcher on inclined surfaces and slopes, vertical transport .....	15
<b>6. Cleaning and care</b> .....	<b>15</b>
<b>7. Maintenance</b> .....	<b>15</b>
<b>8. Storage</b> .....	<b>16</b>
<b>9. Service life</b> .....	<b>16</b>
<b>10. Optional accessories and replacement parts</b> .....	<b>16</b>
<b>11. Identification and Guarantee Certificate</b> .....	<b>25</b>

# ultraBASKET STRETCHER

## 1. PRODUCT DESCRIPTION

### ultraBASKET STRETCHER

#### Technical data:

Product reference:	<b>REF</b> SAN-0087-1
Weight:	12,5 kg
Load capacity:	315 kg
Dimensions (L x W x H):	217 x 61,5 x 18,5 cm

#### Scope of delivery:

- 1 base part (ASA/ABS)
- 1 patient rest
- 1 footrest
- 3 patient restraint straps
- 1 circumferential rope

### ultraBASKET STRETCHER TWIN

#### Technical data:

Product reference:	<b>REF</b> SAN-0087-2
Weight:	15,5 kg
Load capacity:	315 kg
Dimensions (L x W x H):	214 x 61,5 x 18,5 cm
Head part:	116 x 61,5 x 18,5 cm
Feet part:	113 x 61,5 x 18,5 cm

#### Scope of delivery:

- 1 base part (ASA/ABS) split
- 1 patient rest
- 1 footrest
- 2 patient restraint straps
- 1 patient resistant strap with Allen Key
- 1 circumferential rope splitted

### ultraBASKET STRETCHER TWIN-XXL

#### Technical data:

Product reference:	<b>REF</b> SAN-0087-2-XXL
Weight:	36 kg
Load capacity:	400 kg
Dimensions (L x W x H):	214 x 82 x 18,9 cm
Head part:	109 x 82 x 18,9 cm
Feet part:	118,2 x 82 x 18,9 cm

#### Scope of delivery:

- 1 base part (ASA/ABS) split
- 1 patient rest
- 1 footrest
- 3 patient restraint straps
- 1 patient resistant strap with Allen Key
- 1 circumferential rope splitted

### ultraBASKET STRETCHER SMALL

#### Technical data:

Product reference:	<b>REF</b> SAN-0087-1-M
Weight:	12 kg
Load capacity:	300 kg
Dimensions (L x W x H):	218 x 59 x 20 cm

#### Scope of delivery:

- 1 base part (ASA/ABS)
- 1 patient rest
- 1 footrest (M-variant)
- 3 patient restraint straps

## 2. IMPORTANT INFORMATION

Before using the stretcher, please carefully read the instructions for use and safety instructions. Keep the instructions for use in a safe place to avoid operating errors.

### 2.1. The product's intended purpose

The basket stretcher is the medical device and serves to safely transport injured parties. When maneuvering greater height differences, it is used in combination with abseiling equipment.

### 2.2. Customer service

Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealership or ultra-MEDIC GmbH directly. Always specify the LOT-number to identify the product. This number is located on the product identification sticker. This sticker may not be removed.

### 2.3. Declaration of conformity

The basket stretchers were manufactured in accordance with the applicable EC guideline 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

### 2.4. Liability and warranty

The manufacturer grants a warranty period of 24 months on this product. The warranty only encompasses the costs of the spare parts. Any incurred shipping or wage costs are not subject to warranty.

The warranty excludes damages caused by:

- Transport
- Operating errors
- The basket stretcher's incorrect, improper or omitted maintenance
- Damages or breakage caused by the basket stretcher's incorrect operation
- Damage to components with a purely aesthetic function
- Repairs conducted by unauthorized persons
- Use of third-party products
- Damages caused by using the basket stretcher in unsuitable surroundings or under unsuitable conditions
- The usual wear and tear.

### 2.5. User group

According to § 2, Paragraph 2 of the Medical Devices Operator Ordinance (MPBetreibV), this product may only be used by competent, trained and qualified personnel. A prerequisite is theoretical and practical experience in handling the basket stretcher.

## 3. INFORMATION ON THE SAFETY AND PROTECTION OF THE PRODUCT

1. *Before and after every use, check the components and the patient restraint straps for damage and completeness.*
2. *Ensure that the patient restraint straps are correctly assembled.*
3. *Before initial use, conduct a test run with an empty basket stretcher.*
4. *In the event of damage or suspected damage to the straps or the basket stretcher, remove the product from operation immediately until any occurring irregularities have been remedied.*

## 4. INSTALLATION

Article SAN-0087-1, SAN-0087-2 and SAN-0087-2-XXL are delivered with a rope around the circumference of the base part.

**Patient rest:** To attach the patient rest onto the basket stretcher, please use the Velcro straps on the bottom of the base part and the rear of the patient rest.

**Footrest:** Please use the clamping hooks in the recesses of the base part to attach the provided footrest (Refer to Image1). For SAN-0087-1-M please use the carabines to attach the footrest on the rungs of the metal frame.

**Patient restraint straps:** The delivery scope of all basket stretchers includes 3 patient restraint straps (SAN-0087-2-XXL includes 4 pcs.). The patient restraint straps for the models SAN 0087-1, SAN-0087-2 and SAN-0087-2-XXL are attached to the circumferential rope (refer to image 1, 2a / 2b).

In model SAN-0087-1-M, the metal bars in the circumferential metal frame are used for this purpose (refer to image 3a).

**Lifting bridle:** **Anchor points - metal eyelets**  
Models SAN-0087-1, SAN-0087-2 and SAN-0087-2-XXL  
To attach the lifting bridle please use metal eyelets in the base part of the basket stretcher:  
4 anchor points - model SAN-0087-1 and SAN-0087-2 (refer to Image 1, 1a)  
6 anchor points - model SAN-0087-2-XXL

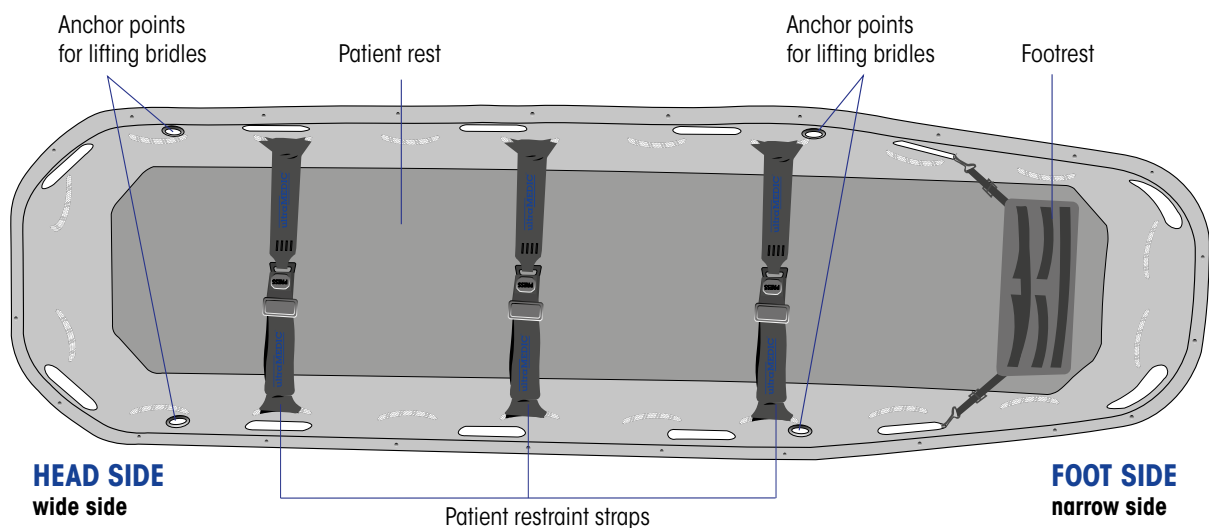
Image 1: Components of basket stretcher and anchor points

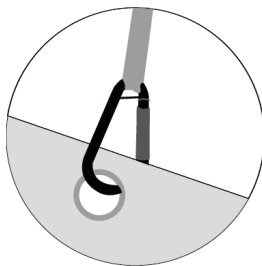
Image 1a: Anchor point in the base part

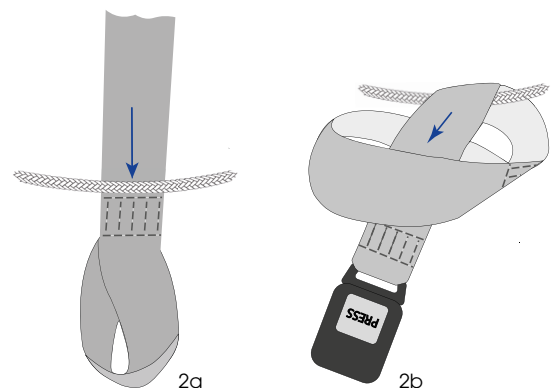
Image 2a/2b: Attachment of patient resistant strap

**Anchor points - metal frame**  
Model SAN-0087-1-M

**Mounting options for lifting bridles:**


1. Directly on the metal frame (Image 3b)

2. With the help of the slings (Image 3c).

The strap slings are attached to the metal frame and the carabiners are attached to the strap slings.

*We recommend mounting via the strap slings, as this prevents the carabiners from moving back and forth between the two bars and avoids jerky changes in position.*

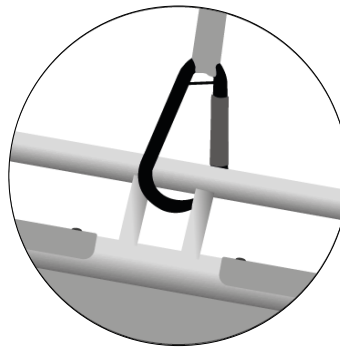**Image 3b:** Mounting directly on the metal frame

**Image 3c:** Mounting with the help of the slings

**Image 3a:** Mounting the patient restraint belts in the SAN-0087-1-M


**4.1. Assembling or dismantling the two-piece basket stretcher**

This section only applies for models SAN-0087-2 and SAN-0087-2- XXL.


1. Firmly push both parts of the basket stretcher together. The metal pins (guide pins) in the head support should disappear completely in the designated openings of the foot support. Ensure that no foreign objects (straps, pebbles, etc.) remain jammed between the head and the foot support.
2. Close the central lock all the way to the stop using the provided Allen key (refer to Image 4). The Allen key is stored in an integrated bag in the patient restraint strap  
*We recommend attaching the patient restraint strap with the Allen key in the centre of the stretcher to allow the stretcher to be assembled/disassembled without removing the strap.*
3. Lock the two external quick-locking devices (refer to Image 5).

**Note!** The quick-locking device can be adjusted, but is already set to the factory default setting. Ensure that the lock is closed but that the base part is not subjected to additional traction.

The basket stretcher is dismantled in reverse order.

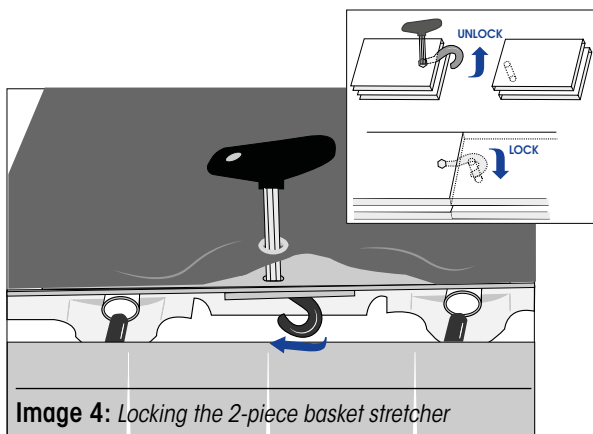**Image 4:** Locking the 2-piece basket stretcher

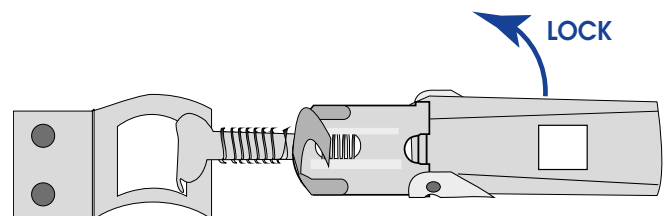**Image 5:** Locking of the external quick-locking devices

## 4.2. Optional accessories: The ultraBELT X-TREME strap system SAN-0089 (not suitable for SAN-0087-1-M)

The special 6-point ultraBELT X-TREME strap was developed for fast and safe patient restraint. It facilitates securing the injured party on the basket stretcher very quickly and also in difficult transport situations.

### 4.3. Attaching the ultraBELT X-TREME strap system to the basket stretcher

The shoulder straps (Image 6-(1)) are attached to the two upper slings (Image 7 - Points A) of the circumferential rope. The two straps also attached to the shoulder straps (Image 6-(2)) can either be used as pelvic straps (attached to the circumferential rope, Image 7 - Points B) or as leg straps (attached to the circumferential rope, Image 7 - Points C). When applying them as pelvic straps, always use an additional straps to fixate the feet. If using them as leg straps, use an additional pelvic strap.

The crotch strap is attached to the loops of the circumferential rope.

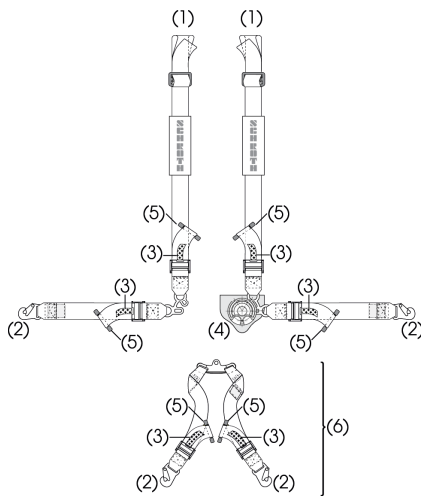
Image 6: Parts of the strap system ultraBELT X-TREME

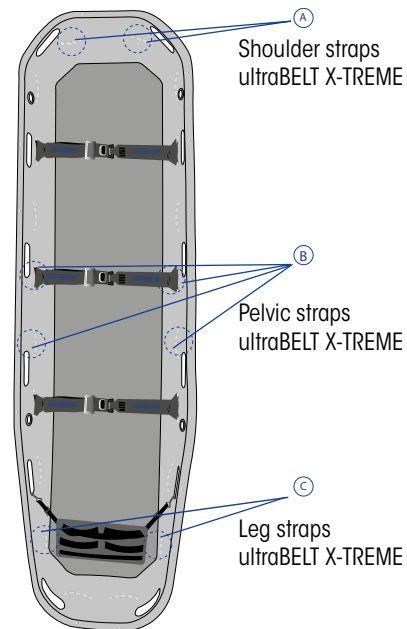
Image 7: Attaching the ultraBELT X-TREME strap system

## 5. USE

### 5.1. Safety instructions

1. A specialist medical assessment must be conducted before the injured party is moved, lifted or transported.
2. At least 2 users are required to operate the basket stretcher to ensure the patient's safety.
3. The injured party must be immobilized on the basket stretcher during every use. Insufficient or no restraint may lead to severe patient or rescuer injuries.
4. Serious incidents that have occurred in connection with the product must be reported by the user and / or patient to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

### 5.2. Positioning the injured party on the basket stretcher


1. Before lifting the patient, bring the basket stretcher as close to the patient as possible and into the most favourable position.
2. Depending on the situation, the patient may have to be repositioned. In this case, remove the injured patient from the previous immobilization system.
3. Restrain the injured party on the basket stretcher using the patient restraint straps. Ensure that the straps are not too tight.

### 5.3. Attaching the footrest

1. Unfold the footrest, carefully slide the lower part under the feet of the injured party and adjust it to a suitable height to stop the injured party from slipping. The feet should be fixated using the two Velcro safety straps (Image 8).

2. Use the clamping hooks to attach the straps of the footrest to the recesses in the basket stretcher (Image 1). For SAN-0087-1-M please use the carabines to attach the footrest on the rungs of the metal frame.

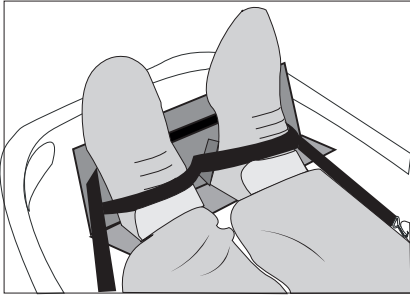**Image 8:** Attaching the footrest

#### 5.4. Operating the ultraBELT X-TREME strap system on the basket stretcher (optional accessories)

1. First use the strap releaser (Image 6-(3)) to loosen all straps and then open the strap using the turn lock (Image 6-(4)).
2. Place both shoulder straps with the pelvic/leg strap aside, position the crotch strap (Image 6-(6)) downwards facing the foot support of the basket stretcher.
3. Place the patient on the basket stretcher
4. Place the two shoulder straps (Image 6-(1)) laterally over the patient, join the halves of the lock and secure them until they engage with an audible sound.
5. Guide the crotch strap from below through the legs of the injured party and also secure it in the belt lock.
6. Fasten the additional pelvic or leg strap.
7. Use the strap tensioner (Image 6-(5)) to tighten all straps. The patient is ready for transport.

#### 5.5. Transporting the basket stretcher on inclined surfaces and slopes, vertical transport

1. Adjust the foot support to a suitable height to prevent the injured party from slipping. If the injured party's lower extremities are injured, immobilize them with suitable additional equipment.
2. Immobilize the injured party using patient restraint straps.
3. Before lifting or lowering the basket stretcher, attach it to a static safety system.

**Note!** Please observe the instructions on how to attach the transport equipment. As optional accessories, we recommend the ultraBRIDLES transport equipment (SAN-0086) from ultraMEDIC. Use third-party transport equipment at your own risk.

## 6. CLEANING AND CARE

*Please do not clean the product using high pressure/steam-jet. This could remove the lubricant and lead to corrosion.*

Use mild soap and warm water to clean the basket stretcher and the straps.

For disinfection, you can use commercially available, alcohol-based agents. Other solutions or cleaning chemicals may damage the material of the basket stretcher and the straps.

## 7. MAINTENANCE

In order to ensure the safety of the user and the injured party, we recommend that you have the basket stretcher and the straps visually inspected by an expert on an **annual** basis.

For German users, we recommend consideration to the following regulations and guidelines:

MPBetreibV  
DGUV Grundsatz 305-002  
DGUV Regel 112-198/199

*In the event of damage or suspected damage, remove the product from operation immediately until any occurring irregularities have been remedied.*

The following parts must be tested to exclude the following defects:

Frame:	Deformation, tears, breaks, broken parts
Base part:	Deformation, tears, breaks, broken parts
Straps:	Cutting and abrasion points, damaged seams, tissue-damaging stains
Footrest:	Deformation, tears, breaks, broken parts
Metal parts:	Rust and oxidation, breaks, tears, deformation
Circumferential rope:	Cutting and abrasion points, too loose, tears, other damages
other damages	

## 8. STORAGE

Keep the basket stretcher in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors could damage the material.

## 9. SERVICE LIFE

The service life of the basket stretcher depends on the individual conditions of use. Depending on the frequency of use, certain wear and tear on the basket stretcher is normal. In this regard, correct use is assumed.

Service life of the patient restraint straps (SAN-0084-S / SAN-0084-SI / SAN-0084-K) is set at max. **10 years**.

The lifecycle of the optional accessories is described in the associated operating instructions.

## 10. OPTIONAL ACCESSORIES AND REPLACEMENT PARTS

SAN-0087-S	ultraFLOATING	Floating collar for all ultraMEDIC rescue stretchers (except for SAN-0087-2-XXL)
SAN-0085	ultraSPLASH GUARD	Splash guard for all ultraMEDIC rescue stretchers (except for SAN-0087-2-XXL)
SAN-0087-SK	ultraSKID	Polyamide runners for attachment to ultraMEDIC basket stretchers frequently pulled over the ground.
SAN-0085-G	ultraGEAR	Undercarriage for attachment to SAN-0087-1 and SAN-0087-2
SAN-0086	ultraBRIDLES	Transport gear, suitable for all models
SAN-0097-P	ultraFLEX PRO	Transport gear, suitable for all models (except for SAN-0087-2-XXL)
SAN-0088	ultraBASKET BAG I	Stretcher and storage bag for SAN-0087-1 and SAN-0087-1-M
SAN-0088-2	ultraBASKET BAG II	Stretcher and storage bag for SAN-0087-2
SAN-0260-2	X-BOON TWO	Vacuum mattress
SAN-0278	ultraVACPUMP I	Vacuum pump for SAN-0260-2
SAN-0087-M	Patient rest	for SAN-0087-1 and SAN-0087-1-M
SAN-0087-M-XXL	Patient rest	for SAN-0087-2-XXL
SAN-0087-M-2	Patient rest	for SAN-0087-2
SAN-0087-F	Replacement footrest	for all models with circumferential rope
SAN-0087-F-M	Replacement footrest	for SAN-0087-1-M
SAN-0087-SE	Circumferential rope	for all models, specify model when ordering
SAN-0084-S	ultraBELT	Replacement patient restraint strap, 1 pcs.
SAN-0084-SI	ultraBELT TWIN	Replacement patient restraint strap with Allen Key, 1 pcs.
SAN-0084-K	ultraBELT Set	Set of replacement patient restraint straps, 3 pcs, contained in a practical storage bag
L-0008-0,2	Strap sling	1 pc, length 20 cm, Load capacity 26 kN

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**ultraMEDIC GmbH**