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
Contents

	<i>Page</i>
1. Product description	10
2. Important information	11
2.1 Liability and warranty	11
2.2 User group	11
2.3 Declaration of conformity	11
3. Information on the safety and protection of the product	11
4. Use	12
4.1 Using the carabiner hooks	12
4.2 Attaching to the basket stretcher	13
4.3 Adjusting the length of the straps	14
5. Cleaning and care	14
6. Maintenance	15
7. Storage	16
8. Service life	16
9. Identification and warranty certificate	25

ultraBRIDLES

1. Product description

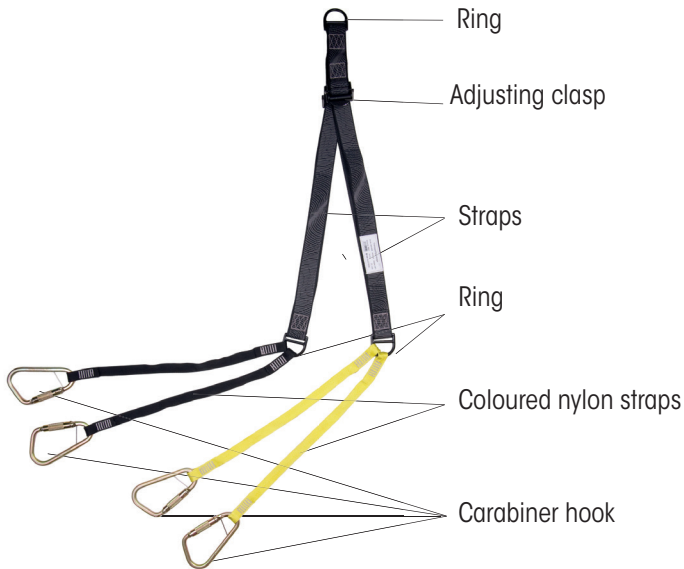Technical data:

Width	Straps top	45 mm
	nylon straps	25 mm
Length	totaly	1600 mm
	nylon straps	570 mm
Length adjustment range per strap		520 mm
D-Ring size		60 x 60 mm
Weight		2,5 kg
Temperature range		-40°C bis +60°C
max. loading capacity		22 kN
Standard / Laws		Medical Devices Regulation EU 2017/745 - MDR
Item-No.		REF SAN-0086

2. Important information

Please read the instructions for use and safety instructions carefully before you start using the ultraBRIDLES. Keep the instructions for use in a safe place to avoid operating errors.

Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealer or ultraMEDIC GmbH directly. Always quote the LOT-number to identify the ultraBRIDLES.

2.1 Liability and warranty

The manufacturer grants a warranty period of 24 months on this product. The warranty only covers the costs of the spare parts. Any incurred shipping or wage costs are not subject to warranty.

2.2 User group

This product may only be used by competent, trained and qualified personnel, and the principles of use must be observed. A prerequisite for use is theoretical and practical experience in handling the ultraBRIDLES.

2.3 Declaration of conformity

This product was manufactured in accordance with the current EU regulation 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

3. Information on the safety and protection of the product

1. Before and after every use, check that the carabiners, straps and claps are intact and complete.
2. Make sure that the ultraBRIDLES is assembled correctly.
3. Before using the ultraBRIDLES for the first time, conduct a test run to familiarize yourself with the product.
4. Observe the principles of use. To prevent the risk of falling, when adjusting the strap make sure that no-one is in a place where there is a risk of falling.

5. In the event of damage or suspected damage to the straps, carabiners, D-rings or clasps, remove the product from operation immediately until any occurring irregularities have been remedied.
6. All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

4. Use

The ultraBRIDLES transporting suspension connects the basket stretcher to the commercially available descenders in order to transport the patient. The length of the ultraBRIDLES transporting suspension can be adjusted.

The ultraBRIDLES is approved for use with the ultraMEDIC rescue stretchers listed below:

Basket stretcher - ultraBASKET STRETCHER - Artnr. SAN-0087-1
Basket stretcher - ultraBASKET STRETCHER TWIN - Artnr. SAN-0087-2
Basket stretcher - ultraBASKET STRETCHER SMALL - Artnr. SAN-0087-1-M
Rescue stretcher - CONREST - Artnr. SAN-9100
Rollable rescue stretcher - ultraROLL STRETCHER - Artnr. SAN-9000
Rollable rescue stretcher - ultraROLL STRETCHER MILITARY - Artnr. SAN-9001

4.1 Using the carabiner hooks

A carabiner hook is attached to each of the four nylon straps to secure the transporting suspension to the basket stretcher.

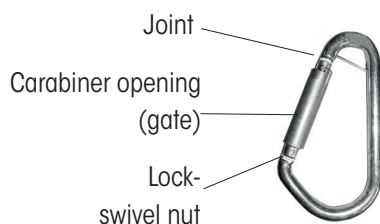
Figure 1: Structure of the carabiner hook

To open the carabiner hook, first turn the lock swivel nut by 1/4 turn to the side. The gate can then be opened, it closes again automatically when released.

To close it, move your thumb off the carabiner gate. This closes the gate, then tighten the lock swivel nut again. This prevents the carabiner hook from being opened accidentally.

4.2 Attaching to the basket stretcher

First, attach the carabiner to the designated eyelet on the basket stretcher and then to the attachment point of the lifting device.

To attach the transporting suspension to the basket stretcher, use the metal eyelets in the plastic trough underneath the main frame (Figure 2).

Figure 2

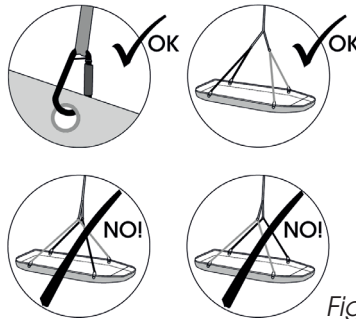
Figure 3

Only the designated eyelets may be used as attachment points on the basket stretcher. Recessed grips on the basket stretcher must never be used for this purpose!

Only use the ring at the top of the ultraBRIDLES as an attachment point. ultraBRIDLES attachment points are exclusively made of metal.

Avoid feeding the straps through the recessed grips as this can change the force distribution and cause increased wear to the straps and seams.

Make sure that the straps are not twisted when attached. This can change the force distribution in the transporting suspension and increase the wear to the straps and seams.

Color-coded pairs of straps for the transporting suspension (e.g. yellow-yellow) can always only be attached in pairs to either the head part or feet part (Figure 3). The other way round does not guarantee the safe guidance of the basket stretcher.

4.3 Adjusting the length of the straps and setting the incline or height of the transporting suspension

The length of the four nylon straps can be adjusted up to 520 mm.

To set the **incline or height** of the transporting suspension, make sure that the top strap is used as the adjustment strap and the bottom strap as the fixing strap. To adjust, undo the clasp and move the strap to the desired position. (FIGURE 2)

To shorten it, hang up the strap. Use one hand to hold the adjustment clasp vertical to the strap. With your other hand, pull the front outside strap down until you have the desired length and then let go. Letting go of the adjustment clasp prevents further adjustment.

Note! The strap may only be pulled forwards to the clasp “nose” when being shortened as this is the only way to set the “excess length”.

To extend it, grip the adjustment clasp of the suspended strap from the back. Pressing on the frame of the clasp releases the lock and the strap slides. The strap will extend under load. Once the desired length is reached, let go of the clasp to reactivate the setting.

In the case of a **vertical rescue**, only 2 pairs of straps of the same color are usually used. The pair of straps that isn't being used can be hooked into the top recessed grips or into the circumferential rope of the rescue stretcher. Make sure though that no load is placed on this pair of straps. Its sole purpose is to secure dangling, loose straps.

5. Cleaning and care

Use mild soap and warm water (40°C) to clean the ultraBRIDLES.

For disinfection, you can use commercially available, alcohol-based agents. Other solutions or cleaning chemicals may damage the material of the straps, carabiner hooks and clasps.

Allow the wet transportation suspension to dry naturally and keep it away from direct heat exposure.

Handwash up to max. 40°

Do not bleach

Do not machine wash
aggressive or tumble dry

Avoid contact with
substances / chemicals

6. Maintenance

The ultraBRIDLES transportation/descender bridle is not reparable.

In order to ensure the safety of the injured party and the user, we recommend that you have the ultraBRIDLES visually inspected by an expert at least once a year.

For users in Germany, we recommend consideration of the following regulations and guidelines:

DGUV Grundsatz 305-002

DGUV Regel 112-198/199

MPBetreibV

For users outside of Germany, regular testing in accordance with country-specific is essential.

The following parts must be tested to exclude the following defects:

Straps	Cutting and abrasion points, damage of the seams, tissue-damaging stains
Metal parts	Rust, oxidation, deformation
Other damage	

In the event of damage or suspected damage, remove the product from operation immediately.

7. Storage

Store the ultraBRIDLES in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors could damage the material.

Dry and store at room temperature.

Store when dry and protected from light.

8. Service life

The discard criteria depends on the product, its frequency of use and the external conditions of use.

The service life of the ultraBRIDLES depends on the individual conditions of use. Depending on the frequency and intensity of use of the transportation suspension, certain wear and tear is normal. Appropriate use is assumed here.

The material is subject to aging even when handled carefully. This is accelerated by UV rays, heat, chemicals, aggressive atmosphere and dirt. The product should be phased-out depending on the condition or at the latest after 6 to 8 years.

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