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CONTENT

1. PRODUCT DESCRIPTION	2
2. INTRODUCTION	25
2.1 Important warnings	25
2.2 Explanation of terms / Consideration	25
2.3 Introduction / Explanation of product	25
2.4 Purpose-conformant use	26
2.5 Usage options	26
2.5.1 Pulling along the ground (even terrain)	26
2.5.2 Transport using the handles	27
2.5.3 Horizontal rescue	27
2.5.4 Vertical rescue	27
2.6 Approvals	27
2.7 Identification marking	27
3. TECHNICAL DATA	28
3.1 Model description	28
3.2 Supplied kit	28
3.3 Optional accessories	29
4. CHECKS BY THE USER	30
4.1 User group	30
4.2 Checks before deployment	30
4.3 Checks after use	30
5. HANDLING THE PRODUCT	30
5.1 Preparation / assembly	31
5.2 Fitting the belts	31
5.3 Assembly	32
6. USE	32
6.1 Safety instructions	32
6.2 Positioning the patient	33

6.3 Horizontal rescue	34
6.4 Winter usage	34
7. CLEANING AND CARE	34
7.1 Cleaning and disinfecting	34
7.2 Drying	34
7.3 Contact with salt water	35
8. STORAGE	35
9. REPAIRS / REPLACEMENT OF COMPONENTS	35
10. MAINTENANCE / INSPECTIONS	35
10.1 Content of the inspections	36
10.1.1 General inspection	36
10.1.2 Functional test and examination of condition	36
10.2 Inspection before first use	37
10.3 Regular inspections:	37
10.3.1 Visual and functional inspection before and after every deployment .	37
10.3.2 Periodically recurring inspections	37
10.4 Extraordinary inspections	38
11. USEFUL LIFE	38
12. CRITERIA FOR WITHDRAWAL FROM SERVICE	38
13. EQUIPMENT GUARANTEE CARD	40
14. INSPECTION CARDS	41-42

2. Introduction

Before first use, familiarise yourself thoroughly with this operating manual. We would like to expressly point out that we do not accept any liability for damage occurring from incorrect use or non-conformance with our instructions / specifications.

Use in deployment is only permitted after the users have familiarised themselves with the correct and safe handling of the equipment. We recommend regularly practising the many possibilities for using this rescue equipment to ensure absolutely safe handling when it is actually deployed.

2.1 Important warnings

For reasons of safety, the ultraROLL-STRETCHER must be withdrawn from use immediately if:

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2.2 Explanation of terms / Considerations

USER: A person who has been trained to handle the equipment.

EXPERT: Person who is familiar with the product with regard to use/maintenance and repairs - This expert makes decisions based on his technical understanding. The expert consultant is responsible for the maintenance and annual inspection.

2.3 Introduction / Explanation of product

The ultraROLL-STRETCHER is a light, universal rescue stretcher that can also be used as a technical rescue unit.

Application: horizontal as well as vertical rescue.

In this context, please ensure the correct choice of the strapping/transport suspension. Only if the strapping is suitably selected are the various rescue situations covered.

The light, compact and robust rescue stretcher,

- approx. 92 cm x 20 cm when rolled out,
 - with fitted braces and footplate approx. 92 cm x 30 cm,
- protects the patient and also facilitates rescues from critical situations.

The ultraROLL-STRETCHER consists of:

- a flexible, rollable duroplastic shell (the resting surface),

- the fixing belts and handles, including the anchoring gear and lanyards.

The ultraROLL-STRETCHER is not envisaged for transporting material that is not used directly for the actual rescue of the patient. Please bear in mind that sharp edges can damage the material of the ultraROLL-STRETCHER.

Only strapping/belts approved by the manufacturer may be used. Self-securing loops of rescuers that serve for protecting the person of the rescuer may only be hooked at a central anchor point – like the rigging plate or central carabiner hook – but not on the equipment itself.

The maximum load - (patient including luggage / accessories) - is 150 kg.

2.4 Purpose-conformant use

The ultraROLL-STRETCHER is to be used exclusively for transporting patients (for example, injured/wounded persons).

Usage temperature: -30°C to +50°C,

loading capacity, max.: 150 kg

Before use, an assessment must always be made, weighing the necessity of transportation and the possibility of further harm to the patient owing to the transportation. The ultraROLL-STRETCHER is envisaged for the following applications:

- On any terrain, especially in terrains hard to access.
- In an urban terrain, even in very narrow spaces as well as pipes.
- Transport via stairs.

However, the „ultraROLL-STRETCHER“ rescue stretcher is not conceived as a spinal rescue device. If a spinal injury is suspected, then before the patient is placed on the ultraROLL-STRETCHER, he or she must be secured in an approved immobilising device, for example, a vacuum mattress.

2.5 Usage options

2.5.1 Pulling along the ground (even terrain)

The patient can be easily pulled on snow or grass. However, bear in mind the increased wear of the resting surface or possible damage from contact with sharp objects.

2.5.2 Transport using the handles

Strong and ergonomic handles, 4 in number on each side

2.5.3 Horizontal rescue

Horizontal rescue is made possible by a special transportation gear.

2.5.4 Vertical rescue

The suspension straps for vertical rescue must be hung in the reinforcement plates.

2.6 Approvals:

The „ultraROLL-STRETCHER“ was tested by TÜV Austria with regard to product safety.

Test report no. FT15-115

The following standards were applied as the basis for the testing procedure, or used for comparison:

- EN 813:2008 Sit harnesses
- EN 1497:2007 Rescue harnesses
- EN354:2010 Lanyards
- EN 358:2000 Belts and lanyards
- EN 12277:2007 Harnesses
- DIN 23400:2008 Rescue stretcher for mining (dragging basket)

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Identification tab:

with model number, article number, date of manufacture: Year - Month

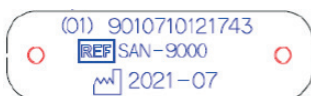
Fig. Example of an identification tab:

The CE mark, general instructions and safety instructions, marking as medical product, batch code (product name with number of the delivery), information for „manufacturer“/“Sold by“ are embossed on the resting surface.

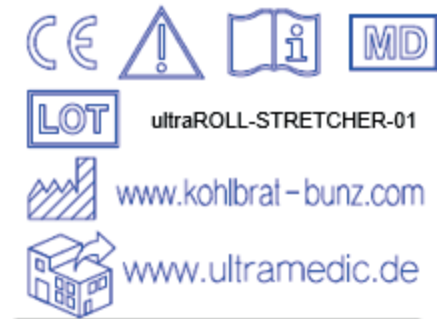
Fig. Identification on the resting surface

3. Technical data

3.1 Model description

ultraROLL-STRETCHER (RL3000)	REF SAN-9000 light red
	REF SAN-9001 olive green
Weight (excluding pocket)	ca. 7,3 kg
Diameter when rolled up	ca. Ø 30 cm
Maximum permissible load	Max. 150 kg
Minimum permissible body dimensions	50 kg / 160 cm
Maximum permissible body dimensions	130 kg / 195 cm
Temperature range for use:	-30°C bis +50°C
Dimensions when assembled:	ca. 200 x 60 x 30 cm
<p>Figure: Ground view Length / width: approx. 254 cm / approx. 92 cm</p> 	

3.2 Supplied kit

- 8 handles (SAN-0260-GR)
- Patient restraining belt with COBRA buckles (SAN-9000-G-1)
- 1 retractor belt (SAN-900-G-3)

- adjustable foot element fixation strap (SAN-9000-G-2A / -2B)
- 1 watertight carrier bag (SAN-9000-T)
- 1 base plate
- 5 protective braces:
 - 1 footrest with 2 clips, 270 x 200 mm,
 - 3 cross-braces:
 - 1 leg protector, reinforcement 390 mm, 690 x 100 mm;
 - 1 pelvis protector, reinforcement 460 mm, 690 x 100 mm;
 - 1 thorax protection, reinforcement 510 mm , 690 x 100 mm;
 - 1 longitudinal brace:
 - 1 spine protection, reinforcement 910 mm, 135 x 1050 mm.

3.3 Optional accessories

Transportation gear:

- Can be used independent of weather changes and in all climatic zones
- Are suitable for winch operation

Basically, only the strapping envisaged for the respective application may be used. There are many options for strapping, depending on the rescue situation. Only the anchoring points provided for the purpose may be used.

Designation	Figure	Catalogue number
ultraBRIDLES Adjustable length transportation gear		SAN-0086
ultraFIX-HORIZONTAL Transportation gear for horizontal rescue		SAN-9005-H
ultraFLEX Transportation gear, adjustable flexibly		SAN-0097
ultraFLEX PRO Transportation gear, adjustable flexibly		SAN-0097-P
ultraFIX-VERTIKAL fixes transportation gear for vertical rescue		SAN-9005-V

Handles must not be used as anchor points or securing points.
Always use the central anchor point as the securing point for the rescuer.

If the original deployment situation changes, then the suitability of the selected strapping should be examined and, if necessary, replaced with another variant.

After direct exposure to sunlight over a prolonged period, the ultraROLL-STRETCHER must not be used in winch operation, and must always be protected from sunlight till it is used as rescue equipment.

4. Tests by the user

4.1 User group

This product may only be used by skilled, trained specialists. Use only by trained persons. Theoretical and practical knowledge of handling the ultraROLL-STRETCHER is a requirement.

4.2 Checks before deployment

The ultraROLL-STRETCHER must be examined before and after every deployment for completeness and readiness for use. A faulty ultraROLL-STRETCHER, or one about which objections have been raised, is not approved for use any more, and must be marked accordingly with a remark about its being blocked for use. The equipment must be immediately handed over to the responsible inspection personnel

Damaged, defective equipment may not be used. Such equipment must be immediately withdrawn from service.

We recommend that after use and inspection, the ultraROLL-STRETCHER should be stowed away, sealed, in the transporting bag. Depending on the internal regulations of the facility, a fresh inspection before use could thus possibly be dispensed with.

4.3 Checks after use

Upon completion of the deployment, the ultraROLL-STRETCHER

- must – if necessary – be cleaned in accordance with the chapter Cleaning and thereafter, inspected for externally visible damage.
- Damaged equipment must be reported to the manufacturer with a description of the defect to check whether repairs are possible.

5. Handling the product

Take the following steps for assembling the ultraROLL-STRETCHER.

5.1 Preparation / assembly of the ultraROLL-STRETCHER

1. Take the contents out of the carrier bag. Loosen the retractor belt. Ensure that all the parts are present, as per the scope of supply.
2. Bend the base plate back so that it can lie flat on the ground.
3. Fit the footrests and the cross-braces if they are not preassembled. When fitting the cross-braces, bear in mind the length differences of the reinforcement. The shortest reinforcement is the first after the assembled footrest, while the longest reinforcement is below the head part.
4. Push the longitudinal brace through below the cross-brace for thorax reinforcement and through the 2 shackles up to the stop. The lower part of the longitudinal brace remains above the cross-brace for pelvis protector (see the figure).

5.2 Fitting the belts

➤ Patient restraining belts

Patient restraining belts are tied in the individual metal-reinforced eyelets on the two sides of the ultraROLL-STRETCHER.

1. Open the buckle and place the halves of the belt on the two sides of the ultraROLL-STRETCHER.
2. Thread the rotated eye of the patient restraining belt into the eyelet.
3. Pull the belt entirely through the rotated eye.

➤ Foot element fixation strap

Ensure that the buckle of the foot element fixation strap is located on the outer side of the ultraROLL-STRETCHER.

1. Take the belt half with the buckle. Thread the rotated eye through the metal-reinforced eyelet and the opening in the plastic shackle.
2. Pull the belt entirely through the rotated eye.
3. Fix the other half of the belt in the nearest metal-reinforced eyelet on the base plate. Lead the end of the belt through the buckle and pull the belt tighter.


➤ Handles

Handles are fixed in the 8 anchor points in the metal-reinforced eyelets. The same eyelets are provided for the handles and the transportation gear.

Both the carrying aids may be fixed and used simultaneously in an eyelet.

➤ Transportation gear

Anchor points for vertical and horizontal suspension are located on both sides of the ultraROLL-STRETCHER.

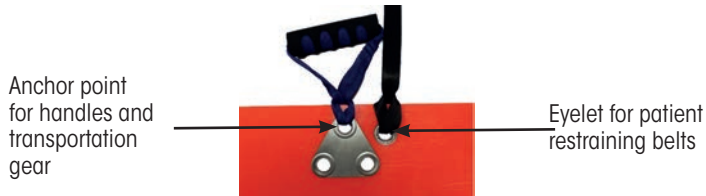5.3 Assembly

1. After the deployment, remove only the spine protector, all the other parts (cross-brace, footrest, belts and handles) can remain assembled.
2. If the ultraROLL-STRETCHER is dirty after use, please read the section: Maintenance / Cleaning in the operating manual.
3. Place the ultraROLL-STRETCHER flat. Start with rolling it up on the head side.
4. Stretch the retractor belt around the rolled-up rescue stretcher.
5. Put all the parts in the carrier bag.

6. Use

6.1 Safety instructions

1. The ultraROLL-STRETCHER is not suitable for catching.
2. An initial medical evaluation must be carried out before the patient is moved, lifted or transported.
3. The ultraROLL-STRETCHER is not a rescue equipment for immobilisation of the spinal column.
4. If there is a suspicion of a spinal column injury of the patient, he or she must first be placed on a spineboard or a vacuum mattress and immobilised.
5. To ensure the safety of the patient, at least 2 persons are required for operating the ultraROLL-STRETCHER.
6. Every time it is used, the patient must be immobilised on the ultraROLL-STRETCHER. Inadequate strapping in, or none at all, can result in serious injuries to the patient and the rescuer.

7. Do not leave the injured person on the stretcher without continuous monitoring under any circumstances.
8. The user and/or the patient must report any serious events that have occurred in conjunction with the product to the responsible authorities of the member State in which the user and/or the patient are resident.

6.2 Positioning the patient

1. Place the ultraROLL-STRETCHER flat next to the patient.
2. Pull the ultraROLL-STRETCHER under the patient. Position the feet below the footplate.

3. Bring the footplate into a vertical position using the foot strap.
4. Fix all the 4 patient securing belts. Ensure that the injured person's breathing is not hindered when the belts are tightened.

5. For small patients, the chest and stomach belts should be strapped cross-wise owing to the danger of strangulation.

Fig. Danger of strangulation with small patients and „normal“ strapping

6. Close the ratchet latch in the head region up to the stop
7. Check all the locks and belts before you lift and transport the injured person.
8. Tighten the foot belt in such a way that the resting surface is firmly in contact at the feet and both the feet are supported between the two belts.

Fig. Foot positioning and tightened foot belt

6.3 Horizontal rescue

1. Retighten the patient belts.
2. Hang the horizontal suspension in accordance with the operating manual of the transportation gear in eyelets provided for the purpose.

6.4 Winter deployment

It is also possible to use the ultraROLL-STRETCHER on snow and ice.

Ensure that the patient is protected against hypothermia with blankets / thermosacks.

Please also note that when the ultraROLL-STRETCHER is used on snow/ice, there is almost no friction between the smooth surface of the ultraROLL-STRETCHER with the snow, and thus, pulling is facilitated.

Moreover, if used on snow/ice, it is necessary to check to what extent additional securing is necessary – „Resecuring during evacuation“.

7. Cleaning and care

7.1 Cleaning and disinfecting

- After every use, the ultraROLL-STRETCHER must be cleaned of foreign bodies by shaking out, removing by hand or brushing off.
- Body fluids etc. can be rinsed out under a water jet, but steamers may not be used. (The water temperature must not be more than 40°C)
- Dirty fabric components can be washed in clear fresh water, if necessary, by adding a standard mild detergent.
- Washing temperature not above 40°C.
- Disinfecting with standard alcohol-based agents. (Follow the usage instructions of the respective manufacturer.)
- Disinfect with pH-neutral disinfecting agents with a quaternary ammonium compound base with chlorhexidine. (Follow the usage instructions of the respective manufacturer.)
- After washing with detergent and/or disinfecting agents, rinse at least twice with clear fresh water to remove all detergent residues.
- Spray-disinfecting with alcohol disinfectants is allowed. (Follow the usage instructions of the respective manufacturer.)

7.2 Drying

- Wet or moist components of the ultraROLL-STRETCHER must be dried in the open air or in well-ventilated rooms.
- No direct sunlight, do not subject to direct heat radiation from heat sources.
- The complete ultraROLL-STRETCHER with its accessories must be taken out of the trans-

porting rucksack and rolled open completely to dry.

- After drying, the entire system must be subjected to a visual and function test.

7.3 Contact with salt water

- An ultraROLL-STRETCHER that is contaminated with salt water or sea water must be kept moist till it is possible to rinse it sufficiently thoroughly with fresh water.
- This rinsing procedure must be repeated at least twice with fresh water. Then, drying.
- Supporting straps of fabric that have been subjected to salt water/sea water for more than 24 hours continuously must be discarded.

8. Storage

- The ultraROLL-STRETCHER must be stored in such a way that its readiness to deploy immediately is unaffected.
- The ultraROLL-STRETCHER should be stored dry, well ventilated and protected from direct sunlight.
- No direct sunlight, do not subject to direct heat radiation from heat sources.
- The storage areas must be kept free of vermin.
- Do not store any fuels, lubricants, acids or chemicals in its immediate vicinity.

ultraROLL-STRETCHERS must not be stored in a wet state.

9. Repairs / replacement of components

Permitted work/modification by the users:

Replacement of belts, belt buckles and carabiner hooks.

10. Maintenance / inspections

The evaluation of the inspection must be documented and preserved.

If the result of an inspection is negative, the ultraROLL-STRETCHER must be sent to the manufacturer for repairs or the equipment must be discarded.

For users in Germany we recommend adherence to the following ordinances and directives:

MPBetreibV (German Medical Device Operation Ordinance)

DGUV Basic Tenet 305-002 (earlier GUV-G 9102)

DGUV Regulation 112-198/199 (earlier BGR/GUV-R 198/199)

Inspection triggers:

- Before first use
- When 12 months have elapsed since the last inspection - even if the ultraROLL-STRETCHER has not been used
- For special reasons, especially after overstressing is determined or if the possibility of overstressing cannot be excluded.
- After malfunctions.
- After damage that has occurred during use.
- After replacement work by users / experts.
- If specially ordered.
- Before/after use.

10.Content of the inspections

10.1.1 General inspection

- Checking to ensure that the odour is not noticeable and neutral (not mouldy or musty)
- Checking for dryness; wet equipment must be dried, any noticeable dirtying of the belts must be cleaned with clean water or a mild detergent and a brush.
- Check all the components (including the operating manual) to ensure that they carry readable identification markings.
- Checking for unusual fouling or discolouring from chemicals or the like.

10.1.2 Functional test and condition examination

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Base plate and base plate shackles	Carabiner hooks on the belts	Belts and handles	Rivet joints
Deformation	Function	Cuts and scouring	Firm seating
Cracks and fractures	Deformation and twisting	Whether the seams are intact	
	Cracks and fractures	Damage	
		Fabric-damaging spots	
		Lapsing of usage duration	

10.2 Inspection before first use

In particular, the following must be checked:

- Completeness
- Dimensional accuracy
- Flawless quality of work
- Correct working of all functional elements
- Absence of damage

10.3 Regular inspections:

10.3.1 Visual and functional inspection before and after every deployment by the user.

- Before and after every deployment, check that the components and the supporting straps are complete and intact.
- Ensure that the patient restraining belts are fitted correctly and the anchor points for the transport gear are firmly in place.
- Before using it the first time, carry out a trial run with an empty ultraROLL-STRETCHER.
- Should there be any damage, or even just suspicion of damage, please immediately withdraw the product from operation till the irregularities that occurred have been rectified.

10.3.2 Periodically recurring inspections:

Every 12 months from the month of first use – evaluation of the condition in accordance with the inspection card - by a qualified person.

Condition as dispatched/condition as delivered	Inspection
After 1 year (date of purchase)	by expert evaluator
After 2 years (date of purchase)	by expert evaluator
After 3 years (date of purchase)	by expert evaluator
After 4 years (date of purchase)	by expert evaluator
After 5 years (date of purchase)	by expert evaluator
After 6 years (date of purchase)	by expert evaluator
After 7 years (date of purchase)	by expert evaluator
After 8 years (date of purchase)	by expert evaluator
After 9 years (date of purchase)	by expert evaluator
After 10 years (date of manufacture)	Maximum useful life reached

10.4 Extraordinary inspections of the ultraROLL-STRETCHER

Extraordinary inspections must be carried out after cases of damage or special events that could affect the carrying capacity, as well as after replacement/repair work.

11. Useful life

Maximum useful life of slinging gear and lanyards:

- 10 years from the month of manufacture

Maximum useful life of the ultraROLL-STRETCHER:

- 10 years from the month of manufacture

Please see the relevant operating manuals for the service life of the optional accessories.

The specified maximum useful life gets reduced by harmful influences like the temperature, mechanical stresses by regular and, if applicable, intensive use, or stress from falling, damage by abrasion, cutting damage, chemicals, fusion owing to high temperature etc.

Depending on the intensity, duration of action and the combination of different influences, these influences can reduce the useful life to different extents.

Intensive use and/or extreme deployment conditions like sharp edges, chemical influences etc. lead to a reduced useful life for reasons of safety. - The user entity must ensure that this is taken into account in the risk analysis of the workplace.

The flawless state and the safe working of the equipment must be confirmed through a periodically recurring inspection, and documented. This inspection must be carried out by a qualified person or expert evaluator. This person must be familiar with the content of the operating manual and should also have understood it.



When the specified useful life has elapsed, the ultraROLL-STRETCHER must be withdrawn from service.

The ultraROLL-STRETCHER must be marked as withdrawn in a clearly visible manner.

12. Criteria for withdrawal from service

The ultraROLL-STRETCHER must be withdrawn from service for horizontal rescues if:

- there are sharp-edged scratches deeper than 1 mm and longer than 50mm
- through-holes or cuts through the material
- there are abrasions scuffing marks of more than 1 mm depth on the resting surface inside the ultraROLL-STRETCHER.
- Overloading/bends/permanent deformation
- Damage that cannot be classified is found

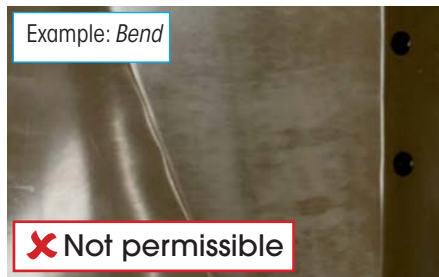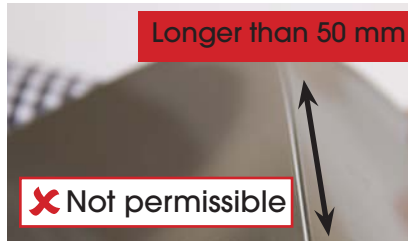Enclosures

13. Equipment guarantee card

1. Manufacturer: Kohlbrat&Bunz GmbH, A-5550 Radstadt, Austria

2. Date of manufacture:

3. Date of 1st use

4. Devices/Parts number:

5. Inspection before first use on: by:

6. Serial number: (to be entered by hand)

Matching of the above data with the label on the relevant equipment is certified by:

Official seal / Stamp of the department / Signature

14. Inspection cards:

INSPECTION CARD FOR ANNUAL MONITORING Part 1

<p>The expert evaluator must completely fill out the inspection card at the time of the periodic annual inspection.</p> <p>This inspection card does not claim completeness as regards the inspection criteria and does not release the expert evaluator from his responsibility regarding his decision about the overall state.</p> <p>Every inspection must be recorded.</p>	Reference number:
	Designation
	Serial number:
	Date of manufacture:

	Date	Signature	Next inspection	Reason for inspection
1st year				
2nd year				
3rd year				
4th year				
5th year				
6th year				
7th year				
8th year				
9th year				
10th year				

INSPECTION CARD FOR ANNUAL MONITORING

Part 2

	1st year	2nd year	3rd year	4th year	5th year	6th year	7th year	8th year	9th year	10th year
Resting surface - no scratches deeper than 1mm?										
Strap undamaged?										
Seam pattern undamaged?										
Strap ends secured / stitched around?										
No deformation of the metal parts?										
Clamping buckles not deformed and freely movable?										
Carabiner hooks present and working properly?										
Usage instructions available										
Product labelling readable										
Miscellaneous:										
In order										
Blocked										

ultraMEDIC[®]

Impressum:

KOHLBRAT & BUNZ GmbH

Loretostrasse 4-8
A-5550 Radstadt

Tel: +43-6452-7193-0

E-Mail: office@kohlbrat-bunz.com

Web: www.kohlbrat-bunz.com

ultraMEDIC GmbH

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Tel.: +49 2631 96983 - 0

Fax: +49 2631 96983 - 33

E-Mail: info@ultramedic.de

Web: www.ultramedic.de