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ultraBELT X-TREME

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INSTRUCTIONS for USE EN

ultraBELT X-TREME

Special 6-Point-Fixing-Strap for basket stretchers

SAN-0089

1. INTENDED USE

The special 6-point ultraBELT X-TREME strap was developed for fast and safe patient restraint. It facilitates securing the injured party on the basket stretcher very quickly and also in difficult transport situations.

2. USER GROUP

This product may only be operated by qualified, expert personnel. A prerequisite is theoretical and practical experience in handling special 6-point-fixing strap.

3. SAFETY INSTRUCTIONS

Never use a strap that is worn, cut or damaged otherwise. Cuts, abrasion and other damage to the strap reduce its protective effect significantly. This can result in trap failure and severe injuries or death. Improper use and the consequences due to the improper use will void the manufacturer's warranty.

4. SECURING THE STRAP SYSTEM IN THE BASKET STRETCHER

- The shoulder straps (1) are attached to the upper two loops (points A) of the circumferential rope (Image 2, 5). To attach the strap, it has to be looped through (Image 4).
- The two straps attached to the shoulder straps (2) can either be used as pelvic straps (attached to the circumferential rope, Points B) or as leg straps (attached to the circumferential rope, Points C). (Image 3, 5).
- When applying them as pelvic straps, always use an additional strap to fixate the feet. If using them as leg straps, use an additional pelvic strap.
- The crotch strap is attached to the loops of the circumferential rope.

5. USING THE STRAP SYSTEM ultraBELT X-TREME

- First use the strap releasers (3) to loosen all straps and open the strap by means of the twist-off cap (Image 1, 6).
- Place the two shoulder straps with pelvic/leg strap aside and position the crotch strap (Image 6) downwards facing the foot end of the basket stretcher.
- Place the patient into the basket stretcher.
- Place the two shoulder straps (Image 6) laterally over the patient. Then the loose strap can be inserted into the strap lock of the other strap.
- Pass the crotch strap from below, between the legs of the injured and also secure it in the strap lock.
- Close the additional pelvic and leg strap.
- Then you can use the strap tensioner (Image 6) to tighten all straps.
- After that the patient is ready for transportation.

Image 1

Image 2

Image 3

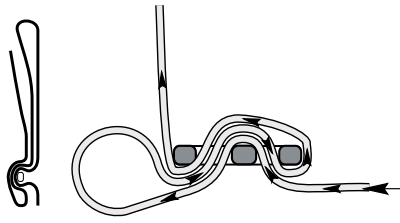
Image 4

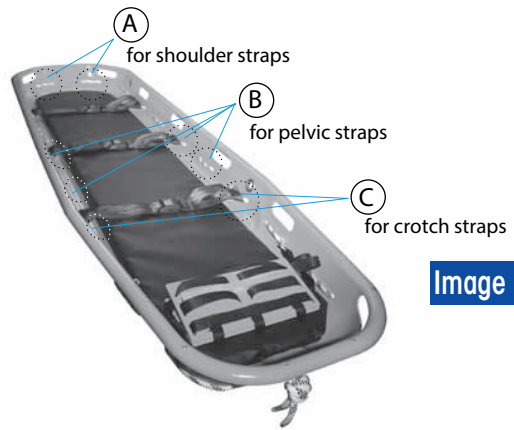
Image 5

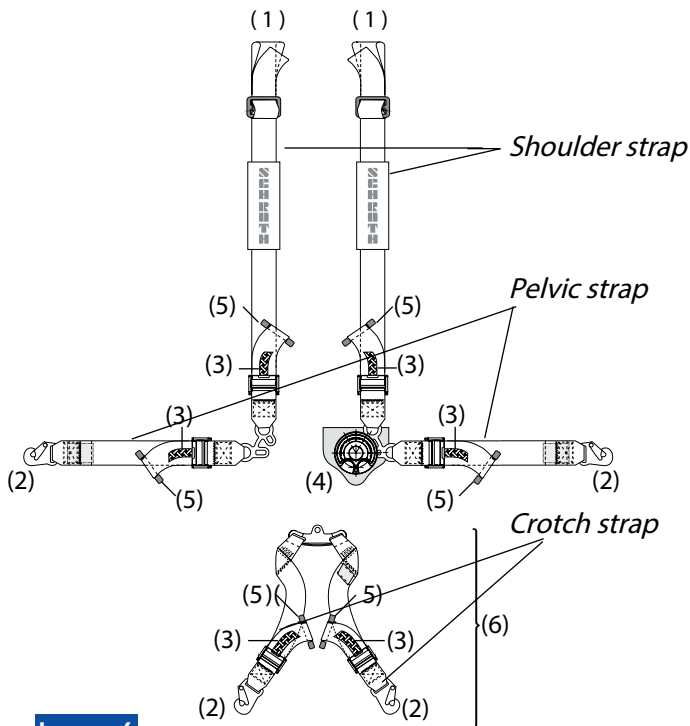
Image 6

- (1) - Shoulder strap
- (2) - Pelvic strap
- (3) - Strap releaser
- (4) - Twist-off cap
- (5) - Strap tensioner
- (6) - Crotch strap

6. CLEANING

- Use only mild soap and warm water to clean the strap.
- Never use solutions or other cleaning chemicals. These can damage the strap or seams.
- Never use chemicals or cleaning solutions to clean the twist-off cap or push-button lock. The very impact-resistant polycarbonate material is sensitive to all types of hydrocarbons, including spray cleaners, which often contain such substances.

7. MAINTENANCE

- Check the strap before each use carefully for damage.
- Never dry the surface in the sun or close to a heater, in a tumble dryer or hair dryer or any other me-chanical or electrical heaters. Strap shrinks when it is heated and the finely tuned stretching proper-ties change.
- Allow to air dry the cleaned strap as usual.

8. STORAGE

Store the product in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors could damage the material.