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Contents

Page

1. Product description	10
2. Important information	11
2.1 Liability and warranty	11
2.2 User group	11
2.3 Declaration of conformity	11
3. Information on the safety and protection of the product	11
4. Intended Purpose and Use	12
4.1 Using the carabiner	12
4.2 Attaching to the rollstretcher	13
5. Cleaning and care	14
6. Maintenance	15
7. Storage	16
8. Service life	16
9. Identification and warranty certificate	17

ultraFIX-HORIZONTAL

1. Product description

Eyelet for attachment to a belay device or an anchor point


- 1** ■ Head end loop strap
- 2** ■ Center section loop strap
- 3** ■ Foot end loop strap

Webbing loop
at the top end
for attaching to
a carabiner

Lashing loop
at the lower end
for attachment to the
anchor point of the
stretcher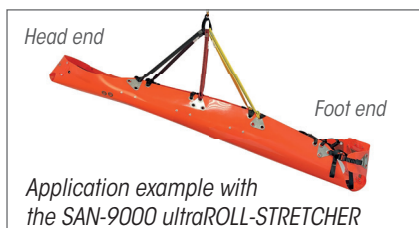

Technical data: :

Width	black loop strap	25 mm
	red loop strap	25 mm
	yellow loop strap	25 mm
Length	black loop strap	650 mm
	red loop strap	600 mm
	yellow loop strap	750 mm
Weight		0,46 kg
Temperature range		-40°C to +60°C
max. loading capacity		20 kN
Standard / Laws		Medical Devices Regulation EU 2017/745 - MDR
Item-No.		REF SAN-9005-H

2. Important information

Please read the instructions for use and safety instructions carefully before you start using the ultraFIX-HORIZONTAL. Keep the instructions for use in a safe place to avoid operating errors.

Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealer or ultraMEDIC GmbH directly. Always quote the LOT-number to identify the ultraFIX-HORIZONTAL.

2.1 Liability and warranty

The manufacturer grants a warranty period of 24 months on this product. The warranty only covers the costs of the spare parts. Any incurred shipping or wage costs are not subject to warranty.

2.2 User group

This product may only be used by competent, trained and qualified personnel, and the principles of use must be observed. A prerequisite for use is theoretical and practical experience in handling the ultraFIX-HORIZONTAL.

2.3 Declaration of conformity

This product was manufactured in accordance with the current EU regulation 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

3. Information on the safety and protection of the product

1. Before and after every use, check that the carabiners and straps are intact and complete.
2. Make sure that the ultraFIX-HORIZONTAL is assembled correctly.
3. Before using the ultraFIX-HORIZONTAL for the first time, conduct a test run to familiarize yourself with the product.
4. Observe the principles of use. To prevent the risk of falling, when adjusting the strap make sure that no-one is in a place where there is a risk of falling.

-
5. In the event of damage or suspected damage to the straps or carabiners, remove the product from operation immediately until any occurring irregularities have been remedied.
 6. All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

4. Intended Purpose and Use

The ultraFIX-HORIZONTAL is a medical device.

Indication: The ultraFIX-HORIZONTAL transport harness provides the connection between the rollstretcher and common rope descent devices in a horizontal position, thus enabling the safe transport of the patient.

Contraindication: No specific contraindications.

The ultraFIX-HORIZONTAL transport harness is not adjustable in length.

The ultraFIX-HORIZONTAL transport harness consists of three loop straps connected by a carabiner for each the left and right carrying side and is used for the horizontal suspension of a rollstretcher.

The Product ultraFIX- HORIZONTAL is approved for use with the ultraMEDIC rescue stretchers listed below:

Rollable rescue stretcher - ultraROLL STRETCHER - Item-No, SAN-9000
Rollable rescue stretcher - ultraROLL STRETCHER MILITARY - Item-No. SAN-9001

4.1 Using the carabiner

To open/close the carabiner, press the locking nut upwards, then turn it a quarter turn to the side. The gate can be opened and will close automatically when released.

To attach the ultraFIX-HORIZONTAL to a descent device or an anchor point, use the eyelets of the carabiners (Figure 1, 1a).

Eyelet for fastening to a
abseiling device or to an
anchor point

Figure 1: Carabiner

Figure 1a: Attachment to a
abseiling device

4.2 Attachment to a rollstretcher

Figure 2: Arrangement and fastening of
the loop strap pairs

Figure 3a: Fastening -
View from the
outside

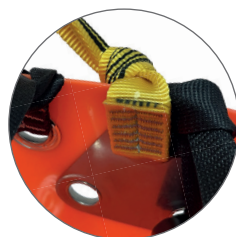
Fig.3a: Fastening -
View from the
inside

1. Open the carabiner.
2. Remove all three loop straps from the carabiner.
3. Attach the loop straps to the left side of the rollstretcher by threading them through the three designated anchor points of the ultraROLL-STRETCHER ('anchor stitch') (see Figures 2, 3ab and 4). Ensure that the striped side of the loop strap faces outwards.
The same anchor points can also be used simultaneously to attach the handles.
4. Thread the free ends of the loop straps back onto the carabiner and close the carabiner (figure 4).
5. Repeat steps 1-4 with the second set on the right side of the ultraROLL-STRETCHER.

Colour-coded loop band pairs of the transport harness must always be attached in pairs (Figures 2, 4):

black - black -> *only in the head section,*
red - red -> *only in the middle section and*
yellow - yellow -> *only in the foot section.*

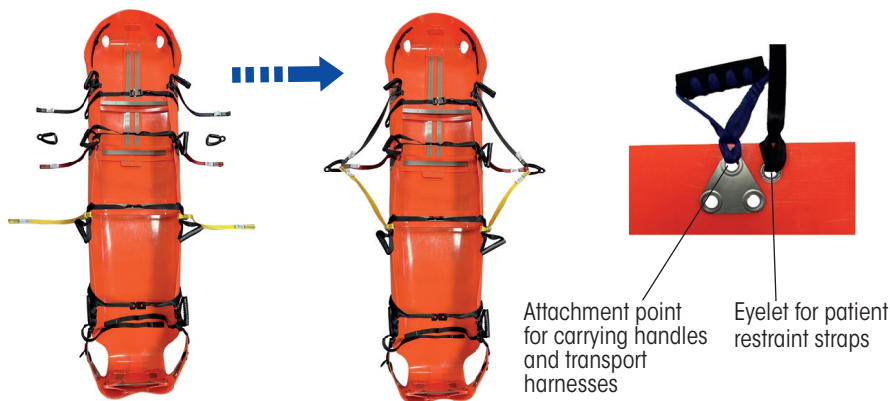
Figure 4: Loop strap attachment

Only the designated eyelets may be used as attachment points on the rollstretcher (Fig.4). **Recessed grips on the rollstretcher must never be used for this purpose!**

At the top, only the eyelets of the two carabiners may be used as attachment points (Fig. 1, 1a). The ultraFIX-HORIZONTAL attachment points are exclusively made of metal.

Make sure that the straps are not twisted when attached. This can change the force distribution in the transporting suspension and increase the wear to the straps and seams.

5. Cleaning and care

Use mild soap and warm water (40°C) to clean the ultraFIX-HORIZONTAL.

For disinfection, you can use commercially available, alcohol-based agents. Other solutions or cleaning chemicals may damage the material of the straps, carabiner hooks and clasps.

Allow the wet transportation suspension to dry naturally and keep it away from direct heat exposure.

Handwash up to max. 40°

Do not bleach

Do not machine wash
aggressive or tumble dry

Avoid contact with
substances / chemicals

6. Maintenance

The ultraFIX-HORIZONTAL transportation/descender bridle is not reparable.

In order to ensure the safety of the injured party and the user, we recommend that you have the ultraFIX-HORIZONTAL visually inspected by an expert at least once a year.

For users in Germany, we recommend consideration of the following regulations and guidelines:

DGUV Grundsatz 305-002

DGUV Regel 112-198/199

MPBetreibV

For users outside of Germany, regular testing in accordance with country-specific is essential.

The following parts must be tested to exclude the following defects:

Straps	Cutting and abrasion points, damage of the seams, tissue-damaging stains
Metal parts	Rust, oxidation, deformation
Other damage	

In the event of damage or suspected damage, remove the product from operation immediately.

7. Storage

Store the ultraFIX-HORIZONTAL in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors could damage the material.

Dry and store at room temperature.

Store when dry and protected from light.

8. Service life

The discard criteria depends on the product, its frequency of use and the external conditions of use.

The service life of the ultraFIX-HORIZONTAL depends on the individual conditions of use. Depending on the frequency and intensity of use of the transportation suspension, certain wear and tear is normal. Appropriate use is assumed here.

The material is subject to aging even when handled carefully. This is accelerated by UV rays, heat, chemicals, aggressive atmosphere and dirt. The product should be phased-out depending on the condition or at the latest after 6 to 8 years from commissioning.

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