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# Contents



*Page*

<b>1. Product description</b> .....	<b>10</b>
<b>2. Important information</b> .....	<b>11</b>
2.1 Liability and warranty .....	11
2.2 User group .....	11
2.3 Declaration of conformity .....	11
<b>3. Information on the safety and protection of the product</b> .....	<b>11</b>
<b>4. Intended Purpose and Use</b> .....	<b>12</b>
4.1 Using the carabiner .....	12
4.2 Attaching to the rollstretcher .....	13
<b>5. Cleaning and care</b> .....	<b>14</b>
<b>6. Maintenance</b> .....	<b>15</b>
<b>7. Storage</b> .....	<b>16</b>
<b>8. Service life</b> .....	<b>16</b>
<b>9. Identification and warranty certificate</b> .....	<b>17</b>

---

# ultraFIX-VERTICAL

## 1. Product description

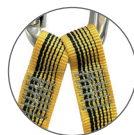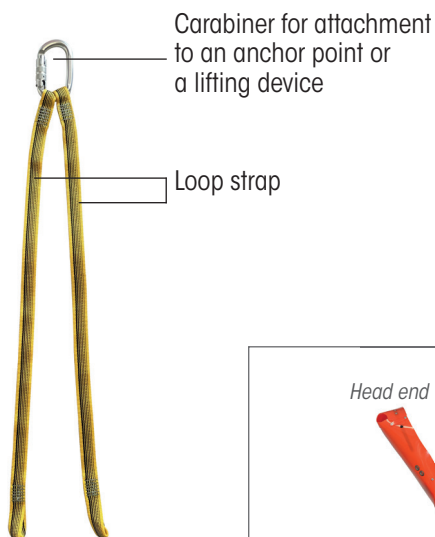**Webbing loop**  
at the top end  
for attaching to a carabiner

**Lashing loop**  
at the lower end  
for attachment to the anchor point of the stretcher

### Technical data :

Width	Loop strap	25 mm
Length	Loop strap	750 mm
Weight		0,17 kg
Temperature range		-40°C to +60°C
max. loading capacity		20 kN
Standard / Laws		Medical Devices Regulation EU 2017/745 - MDR
Item-No.		<b>REF</b> SAN-9005-V

---

## 2. Important information

Please read the instructions for use and safety instructions carefully before you start using the ultraFIX-VERTICAL. Keep the instructions for use in a safe place to avoid operating errors.

Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealer or ultraMEDIC GmbH directly. Always quote the LOT-number to identify the ultraFIX-VERTICAL.

### 2.1 Liability and warranty

The manufacturer grants a warranty period of 24 months on this product. The warranty only covers the costs of the spare parts. Any incurred shipping or wage costs are not subject to warranty.

### 2.2 User group

This product may only be used by competent, trained and qualified personnel, and the principles of use must be observed. A prerequisite for use is theoretical and practical experience in handling the ultraFIX-VERTICAL.

### 2.3 Declaration of conformity

This product was manufactured in accordance with the current EU regulation 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

## 3. Information on the safety and protection of the product

1. Before and after every use, check that the carabiners and straps are intact and complete.
2. Make sure that the ultraFIX-VERTICAL is assembled correctly.
3. Before using the ultraFIX-VERTICAL for the first time, conduct a test run to familiarize yourself with the product.
4. Observe the principles of use. To prevent the risk of falling, when adjusting the strap make sure that no-one is in a place where there is a risk of falling.

- 
5. In the event of damage or suspected damage to the straps or carabiners, remove the product from operation immediately until any occurring irregularities have been remedied.
  6. All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

## 4. Intended Purpose and Use

The ultraFIX-VERTICAL is a medical device.

**Indication:** The ultraFIX-VERTICAL transport harness provides the connection between the roll stretcher and common lifting devices in a vertical position, thus enabling the safe transport of the patient.

**Contraindication:** No specific contraindications.

The ultraFIX-VERTICAL transport harness is not adjustable in length.

The ultraFIX-VERTICAL transport strap consists of two loop straps and a carabiner for the vertical suspension of a rollstretcher.

The Product ultraFIX-VERTICAL is approved for use with the ultraMEDIC rescue stretchers listed below:

Rollable rescue stretcher - <b>ultraROLL STRETCHER</b> - Item-No. <b>SAN-9000</b>
Rollable rescue stretcher - <b>ultraROLL STRETCHER MILITARY</b> - Item-No. <b>SAN-9001</b>

### 4.1 Using the carabiner

**To open/close** the carabiner, press the locking nut upwards, then turn it a quarter turn to the side. The gate can be opened and will close automatically when released.

To attach the ultraFIX-VERTICAL to a lifting device or an anchor point, use the carabiner (Figure 1).

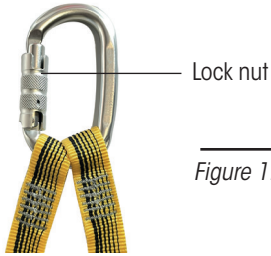
Figure 1: Carabiner

## 4.2 Attachment to a rollstretcher

Fig. 2: Attachment to head-end eyelets

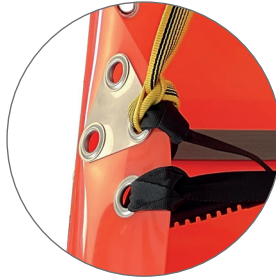
Fig. 3a: Anchor point on the ultraROLL-STRETCHER  
View from the outside

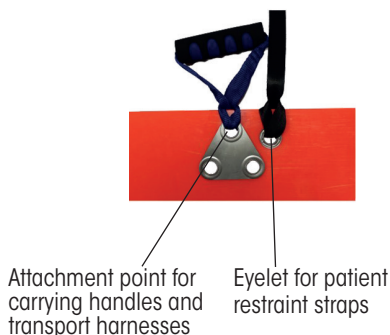
Fig. 3b: Anchor point on the ultraROLL-STRETCHER  
View from the inside

1. Open the carabiner.
2. Remove all three loop straps from the carabiner.
3. Attach the loop straps to the head-end anchor eyes of the rollstretcher by threading them through the eyes of the ultraROLL-STRETCHER ("anchor stitch") (see fig. 2 and 3a-b). Make sure that the striped side of the loop strap is facing outward.

*The same anchor points can also be used simultaneously to attach the handles.*

4. Thread the free ends of the loop straps back onto the carabiner and close the carabiner (figure 2).

*Figure 4: Eyelets of the ultraROLL-STRETCHER*

Only the designated eyelets may be used as attachment points on the rollstretcher (Fig.4). **Recessed grips on the rollstretcher must never be used for this purpose!**

Only the carabiner may be used as the attachment point at the top (Fig. 1). Attachment points of ultraFIX-VERTICAL consist exclusively of metal constructions.

Make sure that the straps are not twisted when attached. This can change the force distribution in the transporting suspension and increase the wear to the straps and seams.

## 5. Cleaning and care

Use mild soap and warm water (40°C) to clean the ultraFIX-VERTICAL.

For disinfection, you can use commercially available, alcohol-based agents. Other solutions or cleaning chemicals may damage the material of the straps, carabiner hooks and clasps.

Allow the wet transportation suspension to dry naturally and keep it away from direct heat exposure.

Handwash up to max. 40°

Do not bleach

Do not machine wash  
aggressive or tumble dry

Avoid contact with  
substances / chemicals

## 6. Maintenance

The ultraFIX-VERTICAL transportation/descender bridle is not repairable.

In order to ensure the safety of the injured party and the user, we recommend that you have the ultraFIX-VERTICAL visually inspected by an expert at least once a year.

For users in Germany, we recommend consideration of the following regulations and guidelines:

DGUV Grundsatz 305-002  
DGUV Regel 112-198/199  
MPBetreibV

For users outside of Germany, regular testing in accordance with country-specific is essential.

### The following parts must be tested to exclude the following defects:

Straps	Cutting and abrasion points, damage of the seams, tissue-damaging stains
Metal parts	Rust, oxidation, deformation
Other damage	

In the event of damage or suspected damage, remove the product from operation immediately.

---

## 7. Storage

Store the ultraFIX-VERTICAL in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors could damage the material.

Dry and store at room temperature.

Store when dry and protected from light.

## 8. Service life

The discard criteria depends on the product, its frequency of use and the external conditions of use.

The service life of the ultraFIX-VERTICAL depends on the individual conditions of use. Depending on the frequency and intensity of use of the transportation suspension, certain wear and tear is normal. Appropriate use is assumed here.

The material is subject to aging even when handled carefully. This is accelerated by UV rays, heat, chemicals, aggressive atmosphere and dirt. The product should be phased-out depending on the condition or at the latest after 6 to 8 years from commissioning.

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