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# Contents



**Page**

<b>1. Product description</b> . . . . .	<b>12</b>
<b>2. Important information</b> . . . . .	<b>13</b>
2.1 Liability and warranty . . . . .	13
2.2 User group . . . . .	13
2.3 Declaration of conformity . . . . .	13
<b>3. Information on the safety and protection of the product</b> . . . . .	<b>13</b>
<b>4. Intended Purpose and Use</b> . . . . .	<b>14</b>
4.1 Using the carabiner hooks . . . . .	14
4.2 Attaching to the basket stretcher . . . . .	15
4.3 Length adjustment of the straps and fixation of the transport sus- pension to adjust the inclination or height of the basket stretcher . . . . .	17
4.3a Length adjustment of the straps . . . . .	17
4.3b Length adjustment of the pulley . . . . .	18
<b>5. Cleaning and care</b> . . . . .	<b>19</b>
<b>6. Maintenance</b> . . . . .	<b>20</b>
<b>7. Storage</b> . . . . .	<b>20</b>
<b>8. Service life</b> . . . . .	<b>20</b>
<b>9. Identification and warranty certificate</b> . . . . .	<b>21</b>

---

# ultraFLEX PRO

## 1. Product description

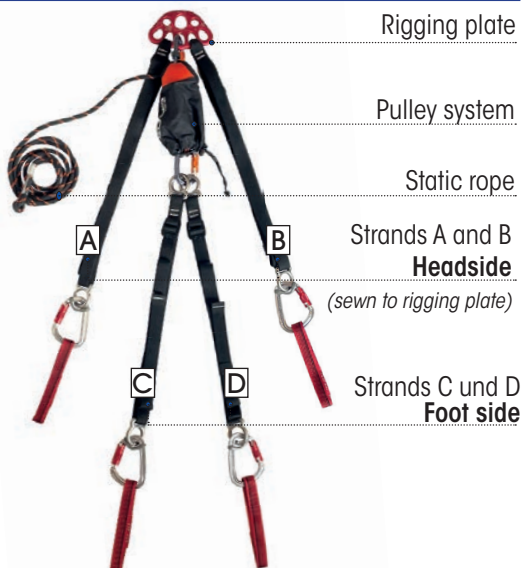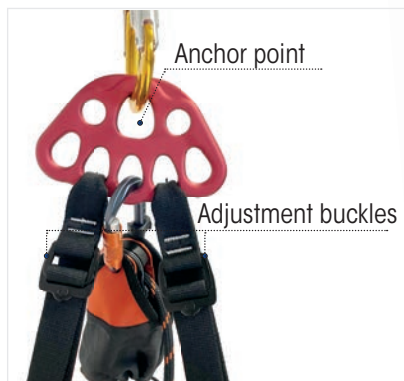### Technical data:

<b>Straps (Width)</b>	<b>Straps Foot side (Strands AB)</b>	30 mm
	<b>Straps Headside</b> (Strands AB - sewn to rigging plate)	30 mm
<b>Straps (Length)</b>	<b>Straps Foot side (Strands CD)</b>	1200 mm
	<b>Straps Headside</b> (Strands AB - sewn to rigging plate)	1500 mm
<b>Minimum Length Adjustment per pair of straps</b>	<b>Straps Foot side (Strands CD)</b>	400 mm
	<b>Straps Headside</b> (Strands AB - sewn to rigging plate)	600 mm
<b>Loops (Length)</b>	-	200 mm
<b>Rigging plate</b>		Plate L 5 Anchor points
<b>Weight</b>		2,5 kg
<b>Temperature range</b>		-5°C bis +60°C
<b>Max. load capacity</b> (please heed max. load capacity of the rescue equipment)		600 kg
<b>Norm / Regulation</b>		MDR: Regulation EU 2017/745 About Medical Devices
<b>Art.-No.</b>		<b>REF SAN-0097-P</b>

---

## 2. Important information

Please read the instructions for use and safety instructions carefully before you start using the ultraFLEX PRO. Keep the instructions for use in a safe place to avoid operating errors.

Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealer or ultraMEDIC GmbH directly. Always quote the lot-number to identify the ultraFLEX PRO.

### 2.1 Liability and warranty

The manufacturer grants a warranty period of 24 months on this product. The warranty only covers the costs of the spare parts. Any incurred shipping or wage costs are not subject to warranty.

### 2.2 User group

This product may only be used by competent, trained and qualified personnel, and the principles of use must be observed. A prerequisite for use is theoretical and practical experience in handling the ultraFLEX PRO and in the field of height rescue.

### 2.3 Declaration of conformity

This product was manufactured in accordance with the applicable EC guideline 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

## 3. Information on the safety and protection of the product

1. Before and after every use, check that the carabiners, straps and claps are intact and complete.
2. Make sure that the ultraFLEX PRO is assembled correctly.
3. Before using the ultraFLEX PRO for the first time, conduct a test run to familiarize yourself with the product.
4. Observe the principles of use. To prevent the risk of falling, when adjusting the strap make sure that no-one is in a place where there is a risk of falling.

5. In the event of damage or suspected damage to the straps, carabiners, rigging plate or clasps, remove the product from operation immediately until any occurring irregularities have been remedied.
6. The ultraFLEX PRO is approved for a maximum system load of 600 kg.

**When using the ultraFLEX PRO with a basket stretcher, it is mandatory to use the supplied loops.**

7. All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

## 4. Intended Purpose and Use

The ultraFLEX PRO is a medical device.

**Indication:** The ultraFLEX PRO transport suspension establishes the connection between the basket stretcher and standard abseiling devices/anchor points on the market to enable the transport of the patient.

**Contraindication:** No specific contraindications.

The ultraFLEX PRO is adjustable in length.

The product ultraFLEX PRO is approved for use with the following ultraMEDIC rescue stretchers:

* Basket stretcher - <b>ultraBASKET STRETCHER</b> - Art.no. <b>SAN-0087-1</b>
* Basket stretcher - <b>ultraBASKET STRETCHER TWIN</b> - Art.no. <b>SAN-0087-2</b>
* Basket stretcher - <b>ultraBASKET STRETCHER SMALL</b> - Art.no. <b>SAN-0087-1-M</b>
** Rescue stretcher - <b>CONREST</b> - Art.no. <b>SAN-9100</b>
** Rollable rescue stretcher - <b>ultraROLL STRETCHER</b> - Art.no. <b>SAN-9000</b>
** Rollable rescue stretcher - <b>ultraROLL STRETCHER MILITARY</b> - Art.no. <b>SAN-9001</b>
* Dragging basket - <b>ultraMINING</b> - Art.no. <b>SAN-0090</b>

\* Attach the transport suspension **only** with the provided webbing loops to the stretcher.

\*\* The carabiners of the ultraFLEX PRO can be directly hooked into the attachment points of the stretcher.

---

## 4.1 Using the carabiner hooks

A ring with a snap hook is attached to each of the four straps to attach the transport suspension to the corresponding stretcher, if necessary using a loop (in combination with a basket stretcher).

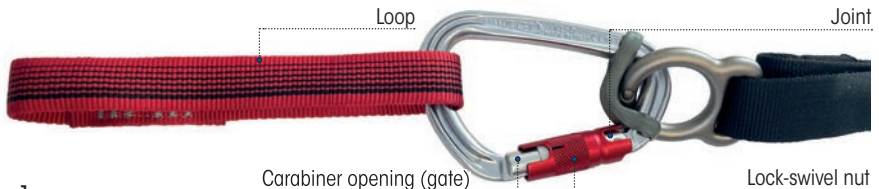
Figure 1

*Structure of the carabiner hook*

**To open** the carabiner hook, first turn the lock swivel nut by 1/4 turn to the side. The gate can then be opened.

**To close**, take your thumb away from the carabiner opening. The fuse closes audibly. This avoids unintentional opening of the snap hook. Make sure that the carabiners are secured.

## 4.2 Attaching to the basket stretcher

1. Before attaching the transport suspension, first detach the pulley system from the central lower eyelet of the rigging plate (Fig. 2).

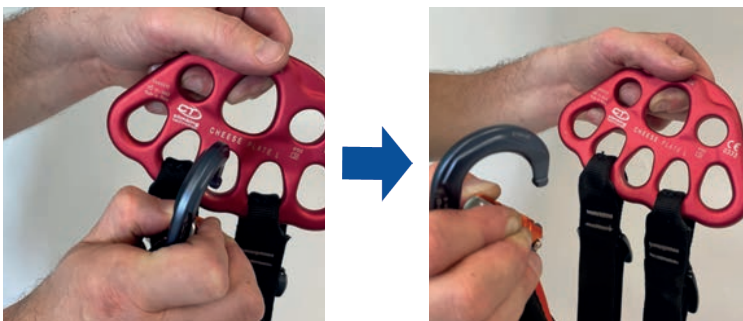
Figure 2

*Detaching the fluff pull system from the rigging plate*

2. 2. Attach the carabiners that originate from the strands A and B of the rigging plate to

---

the corresponding side eyelets in the head area of the basket stretcher using the supplied strap loops. When attaching the carabiners, make sure that the adjustment buckles on the straps A and B are facing outwards (Fig. 3).

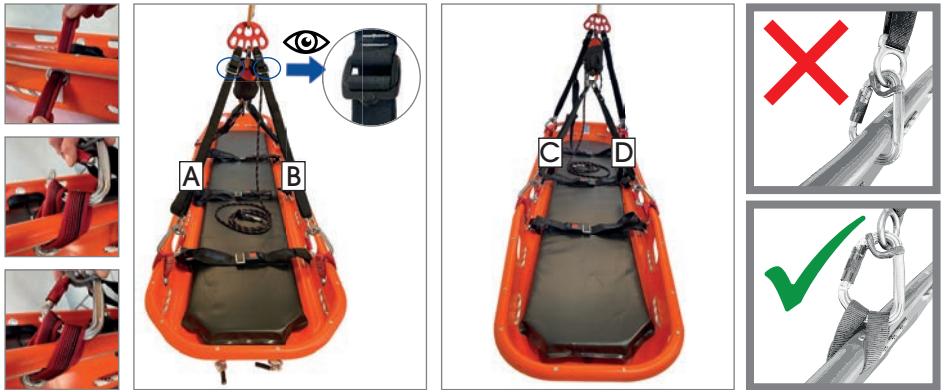
Figure 3

*Attachment of the loops and the straps to the basket stretcher*

3. Attach the carabiners, which originate from the C and D strands of the pulley, to the corresponding side eyelets at the foot of the basket stretcher using the supplied loops (Figs. 3, 4). When attaching the carabiners, make sure that the buckles on the C and D straps are facing outwards (Fig. 3).

Figure 4

*Attachment of the straps on the basket stretcher*

Figure 5

*Fastening the pulley on the rigging plate*

4. Bring both strands together over the basket stretcher and hook the pulley carabiner into the central lower eyelet of the rigging plate (Fig. 5).

5. Hang the central upper eyelet of the rigging plate (anchor point) on the lifting device (Fig. 6).

Figure 6  
Anchor point of the  
lifting device

Only the designated eyelets may be used as attachment points on the basket stretcher. Recessed grips on the basket stretcher must never be used for this purpose!

Only the central upper eyelet of the rigging plate may be used as an anchor point (Fig. 6).

Make sure that the straps are not twisted. This changes the distribution of force in the transport suspension and increases the wear of the straps and seams.

## 4.3 Length adjustment of the straps and fixation of the transport suspension to adjust the inclination or height of the basket stretcher

### 4.3a Length adjustment of the straps

➤ **BASKET STRETCHER** (all ultraBASKET STRETCHER models):

To avoid damaging the basket stretcher under load, the four straps **must NOT BE less than a minimum length of 600 mm (head end) or 400 mm (foot end)**. At the corresponding points on the belts, a red marking, a kind of "stopper", is attached. As a result, the belt cannot be automatically adjusted shorter.

➤ **CONREST and ultraROLL-STRETCHER:**

When using the ultraFLEX PRO in combination with the CONREST or the ultraROLL-STRETCHER, it is allowed to adjust the straps for a shorter time than specified in the instructions for the basket stretcher and to shorten the straps beyond the red stoppers.

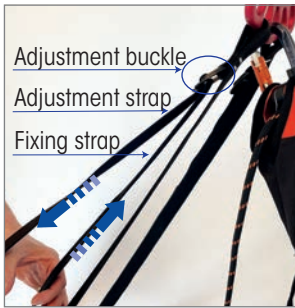
Figure 7

*Adjustment and fixing strap:  
Shortening the straps*

1. To change the **inclination or height** of the transport suspension, make sure that the overhead strap is used as an **adjustment strap** and the middle one as a **fixing strap** (Fig. 7).

**To shorten:** Push the fixing strap up with one hand and at the same time pull the adjustment strap down with the other hand until the desired length is reached, then release. The adjustable buckle prevents further unintentional adjustment (Fig. 7).

Figure 8

*Adjustment and fixing strap:  
Lengthening the straps*

**To extend:** Relieve the fixing strap with one hand and hold the adjustment buckle horizontally to the belt with the other hand. This releases the lock and the webbing can be adjusted by pulling on the fixing strap or will lengthen under the load (Fig. 8). Once the desired length has been reached, let go of the buckle so that the fixation is active again.

### 4.3b Length adjustment of the pulley

The pulley system allows you to adjust the stretcher from the horizontal to the vertical position and back again, even under load. This requires the correct adjustment of the strands.

Figure 9

*Clamping mechanism locked*

The pulley system has a locking system with 2 positions:

#### **Position 1: Clamping mechanism locked.**

In this position, the strap length cannot be extended, but only shortened (Fig. 9).

Figure 10

Clamping mechanism unlocked

### Position 2: Clamping mechanism unlocked.

Before unlocking the clamp, please make sure that the guide rope is held in place, otherwise the outside of the basket stretcher will be quickly lowered.

In this position, the rope length can be extended.

Figure 11

Shortening or lengthening of the Pulley rope

### To shorten:

By pulling on the opposite free end of the rope in the locked position, the rope length can be shortened without the need to re-lock it afterwards (Fig. 11 a).

### To extend:

Unlock the clamping mechanism (position 2) and extend the rope length by gently lowering the stretcher ( Fig. 11 b). Once the desired inclination or rope length has been reached, activate the locking via the clamping mechanism (see position 1).

Avoid jerky relief of the pulley by either applying tension to the loose free end of the rope or securing the stretcher in the foot area.

## 5. Cleaning and care

Use mild soap and warm water (40°C) to clean the ultraFLEX PRO.

For disinfection, you can use commercially available, alcohol-based agents. Other solutions or cleaning chemicals may damage the material of the straps, carabiner hooks and clasps. Allow the wet transportation suspension to dry naturally and keep it away from direct heat exposure.

Handwash up to max. 40°

Do not bleach

Do not machine wash  
or tumble dry

Avoid contact with  
aggressive substances / chemicals

---

## 6. Maintenance

The ultraFLEX PRO is not repairable.

In order to ensure the safety of the injured party and the user, we recommend that you have the ultraFLEX PRO visually inspected by an expert at least once a year.

For users in Germany, we recommend consideration of the following regulations and guidelines:

DGUV Grundsatz 305-002

DGUV Regel 112-198/199

MPBetreibV

For users outside of Germany, regular testing in accordance with country-specific is essential.

**The following parts must be tested to exclude the following defects:**

Straps	Cutting and abrasion points, damage of the seams, tissue-damaging stains
Metal parts	Rust, oxidation, deformation
Other damage	

In the event of damage or suspected damage, remove the product from operation immediately.

## 7. Storage

Store the ultraFLEX PRO in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors could damage the material.

## 8. Service life

The discard criteria depends on the product, its frequency of use and the external conditions of use.

The service life of the ultraFLEX PRO depends on the individual conditions of use.

Depending on the frequency and intensity of use of the transportation suspension, certain wear and tear is normal. Appropriate use is assumed here.

The material is subject to aging even when handled carefully. This is accelerated by UV rays, heat, chemicals, aggressive atmosphere and dirt. The product should be phased-out depending on the condition or at the latest after 6 to 8 years from commissioning.

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